



2020



[www.motranser.com](http://www.motranser.com)

[www.facebook.com/motranser](https://www.facebook.com/motranser)

EN2020.01

2020

# Innovative Functionality

M1 M2 M3 M5 M6 M7

The All-New M-Series  
Functional Dental Mirrors



<https://stomshop.pro>

# Key Advantages of M-Series Mirrors

Motranser Incorporates Functions, Design and Technology Into the M-Series, Completely Reinventing the Dental Mirrors. Each Series is Designed To Meet the Challenges in the Modern Dental Practice. Taking the User-Experience to a New Level.



## UHD Mirror

- Ultra High Definition Mirror, Reflectance = 98%



## Front Surface Alloy Coating

- Multiple Layers of Front Surface Unique Alloy Coating Mirror, Scratch and Chemical Resistant



## Mirror Change

- Patented 1-Click Mirror Replacement System



## Steam Sterilizable

- Sterilization in Class-B Autoclave for More Than 500 Cycles



## AirClean System

- Patented Self AirClean System



## Color Coded

- Color Code Management



## LED Lighting



## 3-Axis Rotation



## Quick Connect



## 3D Suction



## 2 AAA Batteries



## RoHS II Compliant



## CE Certified



## Recyclable

# Bright LED Dental Mirror



- Modular LED Design for Easy Infection Control
- Regulated Circuit for Constant Light Intensity
- Brightness 65,000 Lux White Light
- CRI > 90 to Reveal True Colors

<https://stomshop.pro>

M2

# Rainbow

Dental Mirror with Endo-Guage



6 Colors



- R75 / R98 Anti-Fog Mirror
- Replaceable Mirrors
- Built-In Endo Gauge for Extra Convenience

<https://stomshop.pro>

# Breeze

AirClean Dental Mirror

M3



7 Colors



- Self AirClean System with Hydrophobic Mirrors
- Quick Connect Tubing
- 2 Instant-Air-Connect Options
- 360° Free Rotation Handle

M5

# Mini

AirClean Micro Mirrors



Ø 3mm



Ø 5mm



Ø 15mm



Ø 10mm

4 Colors



4 Sizes

Ø 3 / 5 / 10 / 15 mm



1.6mm



- Design for Use with Microscope or Loupe
- Ultra-Thin 1.6 mm Tip (Available Q2 2020 )
- 4 Mirror Sizes
- Replaceable Tips
- Self AirClean

M6

# Lumi

AirClean Photography Mirrors



Buccal Double-Sided



Lingual Double-Sided




Occlusal(S)



Occlusal(M)



Occlusal(L)

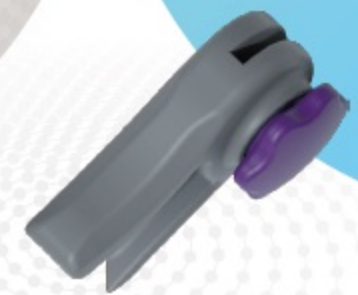

5 Shapes



- R98 Mirrors are Recommended for Professional DSLR Photos
- R75 Mirrors are Recommended for Smartphone Photos
- 5 Mirror Shapes
- Self AirClean

<https://stomshop.pro>



# Eva

## 3D HVE Suction Mirror



Powerful 3D Suction  
20 Suction Holes  
Provide Suction in  
All Perspectives



5 Colors



- Patented 3-Axis Rotation Connector for Easy Maneuverability
- Patented 3D-Suction Provides Powerful Suction in All Directions without Damaging Soft Tissues
- 5.5mm Thin Mirror Tip, the Thinnest in the Industry
- Fits 11mm/16mm Standard Suction Hoses

## Innovative Technologies


Ultra High Definition 98% Reflectance

- Scratch and Chemical Resistant
- RoHS Compliant Materials
- Anti-Fog and Hydrophobic Options



### Mirror Protection Cap

- Most Mirrors Get Damaged During Reprocessing or Storage, The Cap Will Provide Extra Protection to the Mirrors. ( Available Q2 2020)

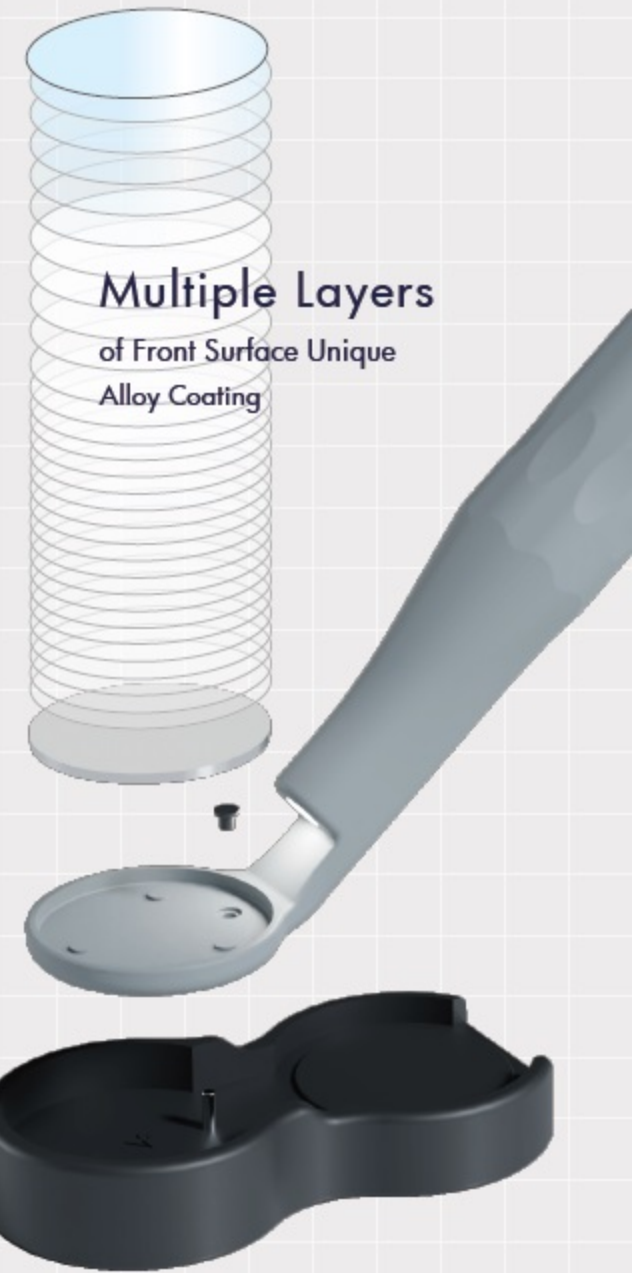

Multiple Layers  
of Front Surface Unique  
Alloy Coating

### 1-Click Mirror Replacement System

- Patented Mirror Change Design
- Replace a New Mirror in Seconds

<https://stomshop.pro>

# The Self AirClean System with Adjustable Air-Flow

Instant Plug-and-Use Air Connections



## Option 1

Get Air from a 3-Way Syringe



## Option 2

Get Air from an Universal 1/4" Female Q/D on Unit

\*The Self AirClean System are Compatible in M3, M5 and M6 Series

## Product Chart

Name	Image	Model
Bright		M1
Rainbow		M2
Breeze		M3
Mini		M5
Lumi		M6
Eva		M7

Description	LED Dental Mirror	Dental Mirror with Endo Gauge	AirClean Dental Mirror	AirClean Micro Mirrors	AirClean Photography Mirrors	3D HVE Suction Mirror
Reflectance	75% / 98%	75% / 98%	75% / 98%	98%	75% / 98%	75% / 98%
Mirror Coating	Front Surface Anti-Fog	Front Surface Anti-Fog	Front Surface Hydrophobic	Front Surface Anti-Fog	Front Surface Anti-Fog	Front Surface Anti-Fog
Size	4.5	4.5	4.5	4 Sizes	5 Shapes	4.5
Diameter	Ø23mm	Ø23mm	Ø23mm	Ø3, 5, 10, 15mm		Ø23mm
Mirror Change	•	•	•	Tip Change		•
AirClean			•	•	•	•
Quick Connect			•	•	•	•
Special Feature	LED White Light	Endo Gauge	M3 & M5 Tips are Compatible	AirClean Mirror Holder	3-Axis Rotation	

<https://stomshop.pro>