mectron

medical technology

IT

EN

DE

FR

ES

PL

easyjet pro

Version of turbine connector

- Unifix® - Unifix L®
- Multiflex® - Multiflex lux®
- Sirona® R - Sirona® B
- Roto Quick® - Roto Quick Lux®
- Borden
- Midwest

CE
0476

Manuale d'uso e manutenzione
Manual of use and maintenance
Gebrauchs- und Wartungshandbuch
Notice d'utilisation et d'entretien
Manual de uso y mantenimiento
Podręcznik użytkowania i konserwacji

Sommario

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Summary

00.0	Introduction.....	21
00.1	Foreword.....	21
00.2	Description of the apparatus.....	21
00.3	Applications.....	22
00.4	Safety requirements.....	22
01.0	Identification data.....	24
01.1	Identification data.....	24
01.2	Equipment identification plate.....	24
02.0	Testing.....	24
02.1	Testing of the equipment.....	24
03.0	Delivery.....	24
03.1	Delivery of the apparatus.....	24
03.2	Lists of material included in the standard supply.....	25
04.0	Installation.....	26
04.1	Safety requirements during installation.....	26
04.2	Connection of the device.....	26
05.0	Use.....	27
05.1	Safety precautions during use.....	27
05.2	Instructions for use.....	29
06.0	Cleaning, disinfection and sterilisation.....	30
06.1	Cleaning and disinfecting the casing of the Easyjet pro.....	30
06.2	Sterilisation procedures.....	30
06.3	Autoclave sterilisation of the front terminal of the polisher.....	31
07.0	Scheduled maintenance.....	31
08.0	Procedures and precautions for disposal.....	32
09.0	Symbols.....	32
10.0	Troubleshooting.....	33
11.0	Technical specifications.....	34
12.0	Warranty.....	35

Unifix[®] - Unifix L[®] is a registered trademark of Bien-Air company
Multiflex[®] - Multiflex lux[®] is a registered trademark of KaVo company
Sirona[®] R - Sirona[®] B is a registered trademark of SIRONA company
Roto Quick[®] - Roto Quick Lux[®] is a registered trademark of W&H company

00.1 Foreword

Before proceeding with the installation, use, maintenance or any other activities on the equipment please read this manual carefully.

Always keep this manual within easy reach.

Important: To avoid causing personal injuries or damage to property, read all the points concerning "Safety requirements" contained in this manual with particular attention.

Depending on the level of risk involved, safety requirements are classed under the following indications:

 **DANGER** (always referred to personal injury)

 **WARNING** (referred to possible damage to property)

The purpose of this manual is to ensure that operators are aware of the safety requirements, of the installation procedures and of the instructions for correct use and maintenance of the apparatus.

The user is not authorised to tamper with the equipment under any circumstances.

If any problems are encountered, please contact a Mectron Service Centre.

Any attempts on the part of the user or any unauthorised personnel to tamper with or alter the apparatus will invalidate the warranty and release the Manufacturers from any liability in respect of any harm or damage to persons or property.

The information and illustrations contained in this manual are up-dated to the date of publication indicated on page 104.

MECTRON are committed to continuous up-dating of their products, which may entail changes to components of the equipment. If there are any discrepancies between the descriptions contained in this manual and your equipment, please contact your dealer or the MECTRON After-Sale service for explanations.

Using this manual for purposes other than those relating to the installation, use and maintenance of the equipment is strictly prohibited.

00.2 Description of the apparatus

The Easyjet pro is a compact bicarbonate-jet whitening system with a quick-fit connection to the turbine.

The Easyjet pro has all the main advantages of the best-known table-top devices:

- a steady and efficient stream of powder
- simplicity of use and reliability.

The operating principle of the Easyjet pro is based on the mechanical action obtained with a jet of crystals of sodium bicarbonate accelerated by a stream of compressed air. The kinetic energy impressed in this way on the particles is almost entirely dissipated as they strike the surface of the enamel, giving rise to a gentle but effective cleansing effect. This action is completed by a jet of water that exploits the depression created around the nozzle to form a bell shape around the main stream, producing two effects: it prevents to a great extent the cloud of bicarbonate from splashing and spilling out and performs continuous rinsing of the area being treated, dissolving the bicarbonate.

00.3 Applications

Bicarbonate-jet polisher. Indications.

- Removal of bacterial plaque;
- Removal of stains caused by tobacco, coffee, tea and chlorhexidine from the surfaces of teeth;
- Prophylaxis in patients undergoing orthodontic treatment.

00.4 Safety requirements

Mectron will not accept any liability for direct or incidental personal injury or damage to property in the following cases:

- 1 If the equipment is used for purposes other than that for which it is intended.
- 2 If the equipment is not used in accordance with all the instructions and requirements described in this manual.
- 3 If the wiring system in the room where the equipment is used does not comply with the applicable standards and appropriate requirements.
- 4 If any assembly operations, extensions, settings, alterations or repairs have been carried out by personnel not authorised by Mectron.
- 5 If the environmental conditions in which the device is kept and stored do not comply with the requirements indicated in the chapter on technical specifications.

DANGER: Qualified and specialised personnel.

The equipment should be used only by specialised personnel having the appropriate training such as dentists and hygienists. The equipment does not produce any side effects if it is correctly used.

DANGER: Intended use.

Use the equipment solely for the purpose for which it is intended (see point "00.3"). Failure to comply with this requirement could lead to serious harm to the patient and/or to the operator and/or damage to/failure of the equipment.

DANGER: Cleaning, disinfection and sterilisation of new or repaired products.

All new or repaired products are delivered in non sterile conditions. Before being used for treatments, all new or repaired products should be cleaned, disinfected and sterilised following the instructions provided under point "06.0" strictly.

DANGER: Infection control.

In order to ensure maximum safety for both the patient and the operator, use only accessories that have been cleaned, disinfected and sterilised. Follow the instructions provided under point "06.0" closely.

DANGER: Use only original Mectron accessories and spare parts.

WARNING: Use only powder manufactured by Mectron (prophylaxis powder).

Do not use powders made by other manufacturers since they could damage the device, give rise to low performance levels and invalidate the warranty covering the device.

DANGER: Check the condition of the device before treatment.

Before each treatment always check that the equipment is in proper working order and that the accessories are efficient. Do not carry out the treatment if any problems are encountered in operating the device. If the problems concern the equipment contact an authorised technical service centre.

⚠ DANGER: Contraindications.

Do not use the device on patients with diseases of the respiratory apparatus, such as chronic bronchitis, asthma, emphysema, etc.

⚠ DANGER: Contraindications.

Patients who use contact lenses should remove them before undergoing treatment with the bicarbonate-jet polisher.

⚠ DANGER: Contraindications.

Do not turn the stream of air/bicarbonate/water in the direction of the eyes.

Failure to observe this requirement can cause serious eye damage. It is advisable for everyone involved (patient, dentist and dental personnel) to wear protective goggles).

⚠ DANGER: Contraindications.

Dentists and other dental personnel are advised to use protective masks in order to limit inhalation of the powder.

⚠ DANGER: Contraindications.

Do not direct the stream of air/bicarbonate/water onto composites, fillings or bridges. Failure to observe this requirement may cause them to be damaged.

⚠ DANGER: Contraindications.

Do not direct the jet of air/bicarbonate/water against the soft tissues or into the gingival groove. Failure to comply with this requirement could cause a tissue emphysema of the gum (mucous membrane and/or subcutaneous emphysema).

01.0 Identification data

01.1 Identification data

An exact description of the model including the serial number of the equipment will make it easier for our After-Sale Service to respond quickly and efficiently to your queries.

Always provide the above information whenever you contact a Mectron Service Centre.

01.2 Equipment identification plate

Each device has an individual serial number (Fig.1 - Ref.A). All the remaining data are contained in this manual (see section "11.0").

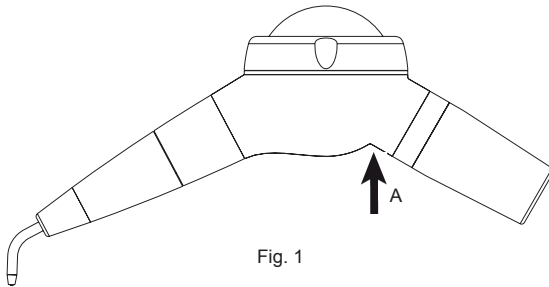
Fig. 1

02.0 Testing

02.1 Testing of the equipment

All equipment manufactured by MECTRON is thoroughly checked and tested, including all components.

During the testing procedure the components are subjected to a number of work cycles.

The tests highlight any malfunctioning due to faulty components.

This procedure ensures proper functioning and reliability of all components.

03.0 Delivery

03.1 Delivery of the apparatus

The equipment contains electronic components that may be damaged by impacts even inside the packaging. Special care must therefore be taken for both transport and storage.

In order to avoid crushing, do not place cartons on top of one.

All material shipped by MECTRON is checked at the time of shipment.

The equipment is delivered properly protected and packaged.

At the time of receipt of the equipment check it for possible transport damage. If any damage is found, make a complaint to the carrier.

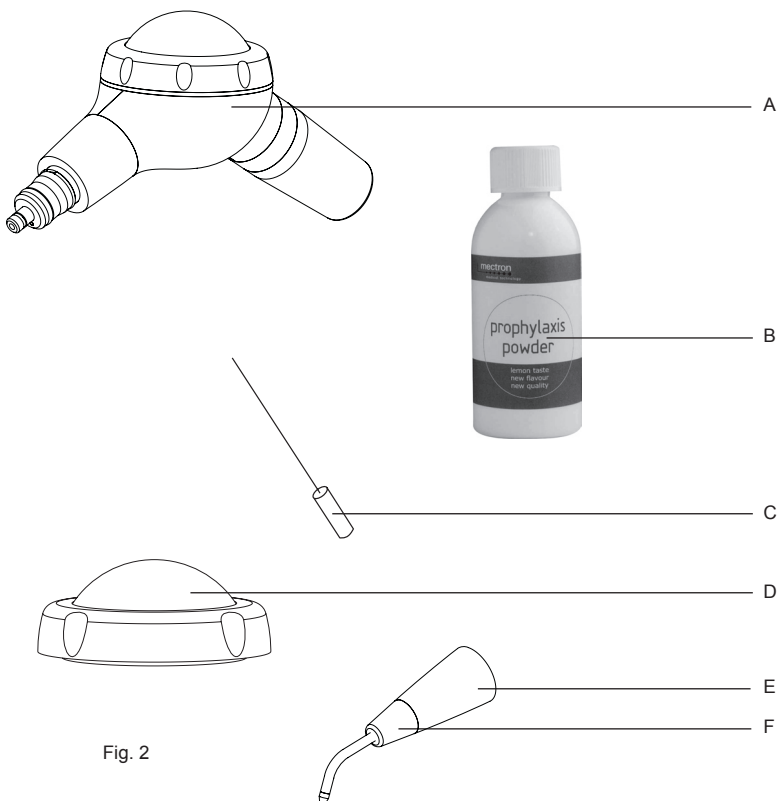
Fig. 2

03.2 Lists of material included in the standard supply

- 1 Body of the apparatus (Fig.2 - Ref.A).
- 1 Pack of prophylaxis powder (Fig.2 - Ref.B);
- 1 Needles for cleaning the front terminal of the polisher (Fig.2 - Ref.C).
- 1 Spare Easyjet pro cap (Fig.2 - Ref.D);
- 1 Jet polisher handpiece head (Fig.2 - Ref.E);
- 1 Front terminal nose cap (Fig.2 - Ref.F).

This equipment may vary at the time of promotional campaigns.

04.1 Safety requirements during installation

 **WARNING:** Do not expose the apparatus to direct sunlight or to sources of UV light.

 **WARNING:** The apparatus is transportable, however it must be handled with care when it is moved.

To ensure that the device is in working order, it is necessary to check the following:

- 1 Supply of water to the turbine spray;
- 2 Supply of air to the turbine;
- 3 Presence of powder;

 **WARNING:** If the supply of water and air is sufficient for the turbine, the Easyjet must work properly.

Standard supplies from the dental unit for a turbine:

- water from 0 to 3 bars (from 0 hPa to 3000 hPa), max. 40°C;
- air 2.3 – 3.2 bars (2300 hPa - 3200 hPa)

04.2 Connection of the device

 **WARNING:** Check that the connector of the turbine is in good condition.

 **WARNING:** Do not connect the Easyjet pro when the turbine is under pressure.

 **WARNING:** Do not press the turbine control pedal while connecting the Easyjet pro.

 **WARNING:** If your turbine has a light, turn the light off.

 **WARNING:** The front polisher terminal and the fitting of the turbine must be perfectly dry. The presence of moisture could cause clogging of the channels in which the air/powder circulate.

The Easyjet pro has a specific adaptor corresponding to the fitting of the turbine on your dental unit. Use the Easyjet pro only for the fitting on your own turbine. Fitting the Easyjet pro to a different connector could damage it.

05.1 Safety precautions during use

DANGER: Contraindications.

Do not use the device on patients with diseases of the respiratory apparatus, such as chronic bronchitis, asthma, emphysema, etc.

DANGER: Contraindications.

Patients who use contact lenses should remove them before undergoing treatment with the bicarbonate-jet polisher.

DANGER: Contraindications.

Do not turn the stream of air/bicarbonate/water in the direction of the eyes.

Failure to observe this requirement can cause serious eye damage. It is advisable for everyone involved (patient, dentist and dental personnel) to wear protective goggles).

DANGER: Contraindications.

Dentists and other dental personnel are advised to use protective masks in order to limit inhalation of the powder.

DANGER: Contraindications.

Do not direct the stream of air/bicarbonate/water onto composites, fillings or bridges. Failure to observe this requirement may cause them to be damaged.

DANGER: Contraindications.

Do not direct the jet of air/bicarbonate/water against the soft tissues or into the gingival groove. Failure to comply with this requirement could cause a tissue emphysema of the gum (mucous membrane and/or subcutaneous emphysema).

DANGER: Check the condition of the device before treatment.

Before each treatment always check that the equipment is in proper working order and that the accessories are efficient. Do not carry out the treatment if any problems are encountered in operating the device. If the problems concern the equipment contact an authorised technical service centre.

DANGER: Infection control and cleaning of the water and air circuits.

In order to ensure maximum safety for both the patient and the operator, after each treatment follow the instructions indicated under point "06.0".

WARNING: Use only powder supplied by Mectron (prophylaxis powder).

Do not use powders made by other manufacturers since they could damage the device, give rise to low performance levels and invalidate the warranty covering the device.

WARNING: Do not attempt to unscrew the cap of the powder container when it is under pressure.

WARNING: Make sure that the powder tank is perfectly dry.

The presence of moisture could cause clogging of the channels.

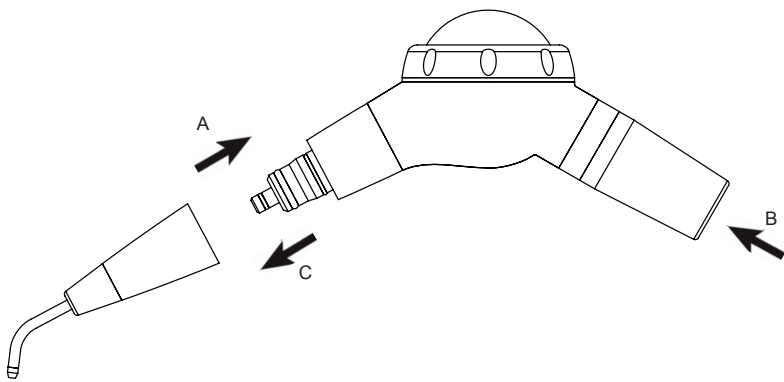
Fig. 3

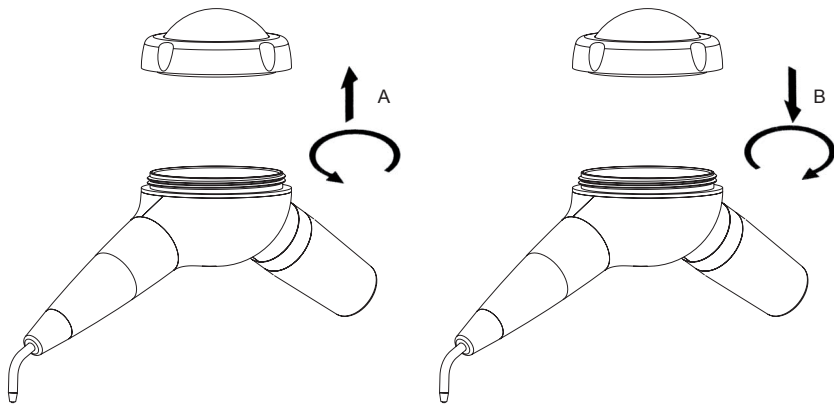
Fig. 4

05.2 Instructions for use

- 1 Insert the front terminal of the polisher (sterile) (Fig.3 - Ref.A) so that it fits closely;
- 2 Connect Easyjet pro to the turbine of the dental unit (Fig.3 - Ref.B);
- 3 Adjust the water flow rate on your dental unit in order to obtain a steady stream of water. It is easier to adjust the flow rate without powder;
- 4 Unscrew the cap of the power container (Fig.4 - Ref.A);
- 5 Pour powder into the container, making sure that the level does not reach the nozzles inside the container;
 - ⚠ **WARNING:** It is advisable to use the measuring cap of the pack of prophylaxis powder, filling it to the correct level for the Easyjet pro.
 - ⚠ **WARNING:** It is advisable not to shake the Easyjet pro after filling it with powder, as this could clog the nozzles.
- 6 Screw the cap of the powder container back on, without tightening it excessively (Fig.4 - Ref.B);
 - ⚠ **WARNING:** Remove any residues of powder from the thread of the cap before screwing it on.
- 7 Press the turbine pedal to carry out the treatment;
 - ⚠ **WARNING:** Use the suction cannula to evacuate the mixture of air, bicarbonate and water from the patient's mouth.
- 8 Carry out the treatment using small circular movements. Direct the nozzle only towards the enamel of the teeth, from the gingival sulcus towards the incisal edge of the tooth, keeping at a distance of between 3 and 5 mm.
- 9 Upon completion, polish all the surfaces of the teeth, setting the water flow rate to the maximum;
 - ⚠ **WARNING:** The stream of air, bicarbonate and water will continue for several seconds after removing your foot from the control pedal. Take these few extra seconds into account when finishing the treatment.
- 10 After the treatment, the treated teeth will be clean and the dental cuticle will have been completely eliminated. It will take the proteins contained in the saliva between two and three hours to form it again. For this lapse of time, the teeth will no longer have their natural protection against stains. The patient must be warned not to smoke and not to eat or drink anything that might stain the teeth (tea, coffee, etc.).

IMPORTANT NOTE: Front terminal of the polisher clogged (no powder coming out).

- 1 Disconnect the front terminal of the polisher from the main body of the Easyjet pro (Fig.3 - Ref.C).
- 2 Use the cleaning needle supplied with the device to free the channel from any residues of powder (Fig.2 - Ref.C), carrying out the movements indicated (Fig.5). Blow compressed air into the central hole of the terminal from both ends, using the syringe.
 - ⚠ **WARNING: Clean the channel of the nozzle using only the cleaning needle supplied with the device (Fig.2 - Ref.D).**
- 3 Also clean the front connecting part of the casing of the Easyjet pro thoroughly. If necessary, remove any residues of powder by blowing compressed air onto them with the syringe.
- 4 Connect the front terminal of the polisher to the Easyjet pro (Fig.3 - Ref.A).

06.0 Cleaning, disinfection and sterilisation

06.1 Cleaning and disinfecting the casing of the Easyjet pro

⚠ DANGER: Disconnect the Easyjet pro from the turbine.

Always disconnect the Easyjet pro from the turbine before carrying out any cleaning, disinfection and sterilisation activities.

⚠ DANGER: The Easyjet pro is not protected against the penetration of liquids. Do not dip it into any liquids and do not spray liquids directly onto the casing of the device.

⚠ DANGER: The Easyjet pro is not sterilisable.

After each treatment, proceed as follows:

- 1 Disconnect the front terminal of the polisher from the main body of the Easyjet pro (Fig.3 - Ref.C).
- 2 Remove any residues of powder from the Easyjet pro container.
- 3 Clean and disinfect the surfaces using a cloth moistened with a mild detergent or disinfectant having a neutral pH (pH7), paying special attention to the threaded part of the front cavity. If necessary, remove any residues of power by blowing compressed air on them with the syringe.

⚠ WARNING: Follow the instructions provided by the manufacturer of the disinfectant carefully.

NOTE: Water-based disinfectant solutions with a neutral pH are strongly recommended. Some alcohol-based disinfectants may be harmful and discolour or damage plastic materials. Detergents in powder form may damage the surfaces of Easyjet pro.

06.2 Sterilisation procedures

⚠ WARNING: The only method to be used for sterilising the device is the steam autoclave method.

Do not use any other sterilisation procedures (dry heat, radiation, ethylene oxide, gas, low-temperature plasma, etc.).

⚠ DANGER: The Easyjet pro is not sterilisable.

⚠ DANGER: Infection control - Sterilisable parts - Remove all residues thoroughly before sterilising.

To avoid infections caused by bacteria or viruses, always clean, disinfect and sterilise the following components after each treatment:

- 1 Jet polisher handpiece head (Fig.2 - Ref.E).
- 2 Needle for cleaning the front terminal of the polisher (Fig.2 - Ref.C).

These components are made of materials able to withstand a maximum temperature of 135°C for a maximum of 20 minutes.

The autoclave sterilisation process must be carried out using either of the following parameters:

- Cycle at 121°C for 16 minutes;
- Cycle at 134°C for 4 minutes.

All the stages of sterilisation must be carried out by the operator in accordance with EN ISO 17665-1:2006 and EN 556-1:2001 standards.

06.3 Autoclave sterilisation of the front terminal of the polisher

- 1 Disconnect the front terminal of the polisher from the body of the Easyjet pro (Fig.3 - Ref.C).
- 2 Free the channel from any residues of powder using the cleaning needle included in the supply (Fig.2 - Ref.C) carrying out the movements indicated (Fig.5). Blow compressed air into the central hole of the terminal from both sides with the syringe.

⚠ WARNING: Use only the cleaning needle supplied with the apparatus to clean the channel of the nozzle (Fig.2 - Ref.C).
- 3 **⚠ WARNING:** If the water is too hard, this may cause the spray of water to be too weak. To eliminate thus problem, dipping the front terminal of the polisher in a 2 % solution of hydrochloric acid for a few minutes is recommended.

⚠ WARNING: Rinse the terminal thoroughly in distilled water to eliminate all the residues of the hydrochloric acid solution. Dry it by blowing compressed air into the central hole of the terminal from both sides with the syringe.
- 4 Disinfect the terminal using a mild disinfectant solution with a neutral pH. Dry it by blowing compressed air into the central hole of the terminal from both sides with the syringe.
- 5 Insert the cleaning needle into the channel in the front terminal and seal both items inside a disposable bag.
- 6 Sterilise the bag containing the front terminal of the polisher and the cleaning needle.

⚠ WARNING: Blow compressed air into the front terminal with the syringe before using it again, to make sure that no residual mixture is left inside it.

07.0 Scheduled maintenance

- 1 If the air passage in the front terminal gets clogged, it is sufficient to switch on the device for a few seconds and to use the needles included in the supply (Fig. 5) to free the channel from any residues of powder.
- 2 If the front terminal gets clogged due to the excessively hard water, it is recommended to dip it in a 2 % solution of hydrochloric acid for a few minutes.
- 3 Whenever the front terminal is removed, it is necessary to blow compressed air with the syringe, both into the cavity of the handpiece connector and into the rear central hole of the front terminal, so as to avoid the accumulation of residues of powder.
- 4 To avoid clogging, it is advisable to empty the powder container of the Easyjet pro at least once a day.

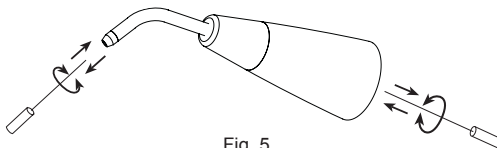
Fig. 5

08.0 Procedures and precautions for disposal

The procedures and precautions to be observed for disposing of the apparatus are those indicated in the current regulations for any other electronic apparatus that is no longer in use.

09.0 Symbols

N.B.: Please read carefully the instructions for use.

0476

Apparatus in accordance with EC Directive 93/42 EEC
Notified body: CERMET.

10.0 Troubleshooting

EN

If the apparatus does not seem to be working correctly, read the instructions again and then check the following table.

PROBLEM	POSSIBLE CAUSE	SOLUTION
The cap of the powder container cannot be unscrewed.	The Easyjet pro is connected to the turbine and the powder container is pressurised.	Disconnect the Easyjet pro from the turbine so as to depressurise the internal circuit.
No powder comes out of the front terminal of the polisher during operation.	Dental unit not connected to the air circuit.	Check the connection between the dental unit and the air circuit.
	The tank of the Easyjet pro is empty.	Read point "05.2".
	Front terminal polisher clogged due to excessive moisture in the powder or insufficient cleaning/maintenance.	Read point "06.1" and "06.3". Remove the powder from the container and clean it with a dry cloth. Read point "07.0"
	The level of powder inside the container is higher than permitted.	Restore the correct level of powder in the container (see point "05.2").
	Unsuitable powder.	To enable the apparatus to work correctly use only powder produced by MECTRON.
No water comes out of the front terminal of the polisher during operation.	Dental unit not connected to the water circuit.	Check the connection between the dental unit and the water circuit.
	Water too hard.	Read point "06.3 - 07.0".
	Water filter of dental unit clogged.	Check the connection to the water circuit of the dental unit.

PROBLEM	POSSIBLE CAUSE	SOLUTION
Leakage of powder from the cap of the container.	Cap not correctly screwed in place.	Screw the cap on correctly.
	Residues of powder in the thread.	Read point "06.1"
Treatment ineffective	Insufficient pressure of the air-supply circuit.	Check the pressure in the air circuit of the dental unit.
	Level of powder inside the container too low or too high.	Restore the correct level of the powder inside the container (see point "05.2").
	Unsuitable powder.	To enable the apparatus to work correctly use only powder produced by MECTRON.
	Front terminal of the polisher clogged by too much moisture in the powder or insufficient cleaning/maintenance.	Read point "06.1" and "06.3". Remove the powder from the container and clean it with a dry cloth. Read point "07.0".

11.0 Technical specifications

Apparatus in accordance with EEC Directive 93/42: Class II a.

Apparatus for intermittent operation: 60" ON 30" OFF.

Standard water pressure provided by the dental unit for a turbine:

From 0 to 3 bars (from 0 hPa to 3000 hPa), max 40°C.

Standard air pressure provided by the dental unit for a turbine:

From 2.3 to 3.2 bars (2300 hPa - 3200 hPa).

Operating conditions:

from +10°C to +40°C.
Relative humidity from 30% to 75%.

Transport and storage conditions:

from -10°C to +70°C.
Relative humidity from 10% to 90%.
Air pressure P: 500hPa/1060hPa.

Weight and size:

125 g.

Before being placed on the market, all MECTRON equipment undergoes a thorough final check to ensure that it is in proper working order.

MECTRON warrant their products, purchased brand-new from authorised MECTRON dealers or importers, free from material or manufacturing defects for a period of 1 (ONE) year from the date of purchase.

Throughout the warranty period, MECTRON undertake to repair (or, at their sole discretion, to replace) free of charge any parts that, in their opinion, are faulty.

Complete replacement of MECTRON products is excluded.

Mectron cannot accept any liability for direct or incidental damage or personal injury in the following cases:

- If the equipment is used for purposes other than that for which it is intended.
- If the equipment is not used in accordance with all the instructions and requirements described in this manual.
- If the wiring system in the room where the equipment is used does not comply with the applicable standards and appropriate requirements.
- If the wiring system in the room where the equipment is used does not comply with the applicable standards and appropriate requirements.
- If any assembly operations, extensions, settings, alterations or repairs have been carried out by personnel not authorised by Mectron.
- If the environmental conditions in which the device is kept and stored do not comply with the requirements indicated in the chapter on technical specifications.

Accidental damages due to transport, incorrect use or carelessness or to connection to power supplies other than as envisaged and damage to the signalling lamps, handpieces and all accessories are excluded from the warranty.

The warranty will no longer apply if the apparatus has been tampered with or repaired by unauthorised personnel.

WARNING

The warranty is valid only if the warranty slip enclosed with the product has been completed in full and returned to us or, if appropriate, to your MECTRON dealer or importer within 20 (TWENTY) DAYS from the date of purchase, as proven by the consignment note/invoice issued by the dealer/importer. In order to benefit from the warranty service, the customer must return the apparatus to be repaired to the MECTRON dealer/importer from which it was purchased, at his own expense.

The apparatus should be returned suitably packed (possibly in its original packing material), accompanied by all the accessories and by the following information:

- a) Owner's details, including his telephone number.
- b) Details of the dealer/importer
- c) Photocopy of the consignment note/purchase invoice of the apparatus issued to the owner and indicating, in addition to the date, also the name of the apparatus and its serial number.
- d) A description of the problem.

Transport and any damages caused during transport are not covered by the warranty.

In the event of failures due to accidents or improper use, or if the warranty has lapsed, repairs to MECTRON products will be charged on the basis of the actual cost of the materials and labour required for such repairs.

The information given in this manual is not binding and can be modified without prior notice.

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Unifix® - Unifix L® is a registered trademark of Bien-Air company
Multiflex® - Multiflex lux® is a registered trademark of KaVo company
Sirona® R - Sirona® B is a registered trademark of SIRONA company
Roto Quick® - Roto Quick Lux® is a registered trademark of W&H company

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Mectron S.p.A.