

## Piezon 150 / Piezon 250 Spare parts replacement under warranty

Valid until Dec. 2014

### Principle

- 1- The first year after the product launch, EMS will limit the available spare parts.
- 2- To replace a defective product under warranty, the customer has to contact EMS Nyon.
- 3- It is mandatory to ship the defective product to Nyon before receiving the warranty replacement.
- 4- There is neither specific service manual nor technical training the first year after the product launch.
- 5- The available spare parts can be changed without any technical skills.
- 6- If the failure is located inside the control unit, we propose to replace the unit itself.


### Contacts

- 1- To contact the technical service, for a warranty claim, please send a mail to: techsupport@ems-ch.com .
- 2- Address

To ship a device, please use the following address:

**EMS - Electro Medical Systems**  
 Technical Service  
 Route de Champ Colin 18  
 1260 - NYON  
 Switzerland

### Drawings of available spare parts

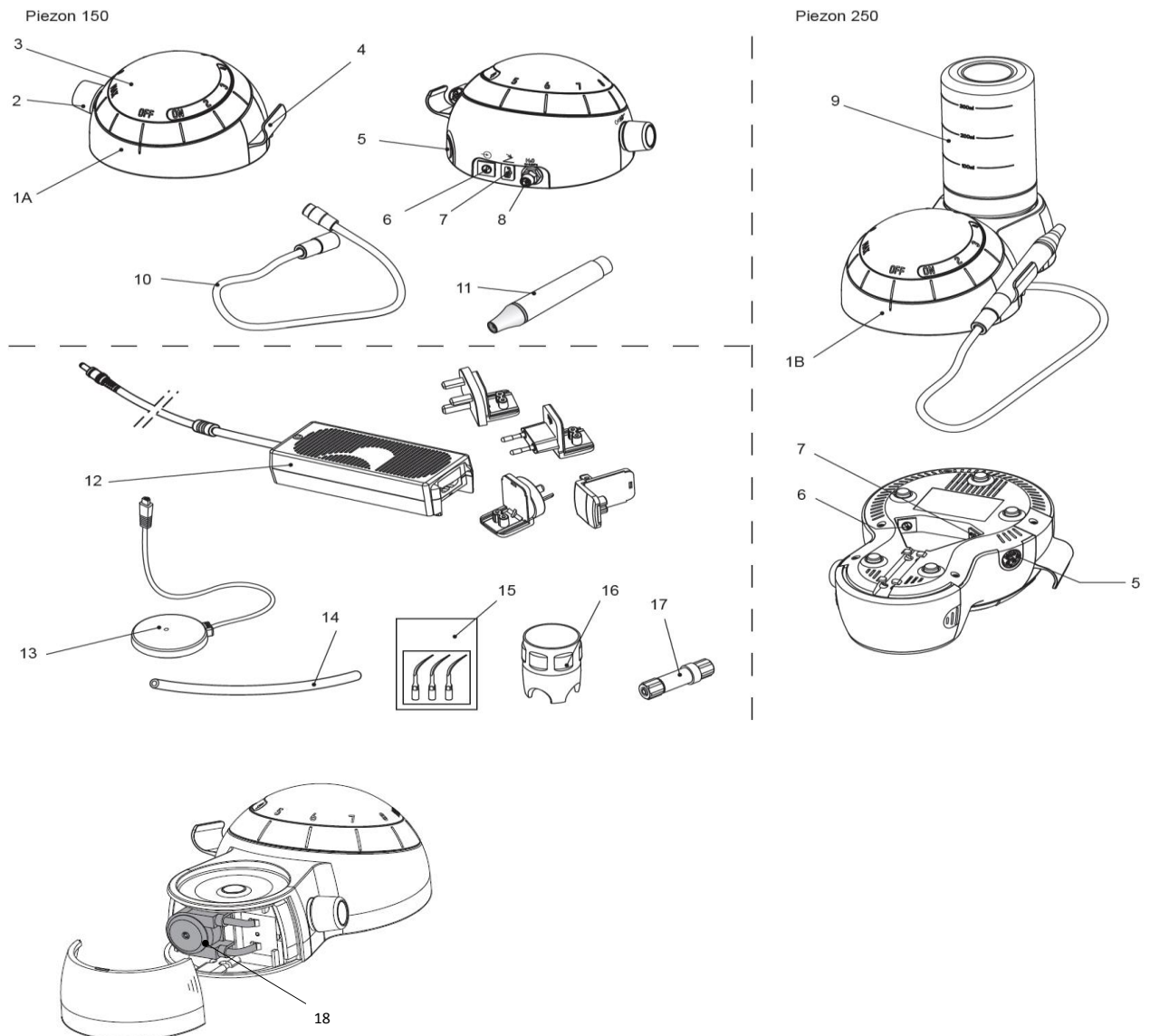

# Piezon 150 / Piezon 250

## Spare parts replacement under warranty

Valid until Dec. 2014

### Drawings of available spare parts

Number in the drawing	Description of defective spare parts	Ref #
1A	Piezon 150 control Unit	FT-215W
	O'ring for cord connection (x2)	BC-229
1B	Piezon 250 control Unit	FT-216W
	O'ring for cord connection (x2)	BC-229
2	Button	FT-215W or FT-216W
3	Cover	FT-215W or FT-216W
4	Handpiece holder	AB-422/A
5	Cord connector on unit	FT-215W or FT-216W
6	Power supply connector	FT-215W or FT-216W
7	Footswitch connector	FT-215W or FT-216W
8	Water supply connector	FT-215W or FT-216W
9	Bottle	EG-064A
10	Cord	EM-140/A
	O'ring for Handpiece connection (x2)	BC-229
11	Handpiece	EN-060/A or EN-061/A
	Light guide	FV-065
	Nose	AD-644
	Flat blue ring (for EN-061/A)	AB-348/C
	Flat red ring (for EN-060/A)	AB-348/B
12	Power supply	EK-357/A
13	Footswitch	EK-128
14	Water hose (for Piezon 150)	FV-090
15	Tips	DS-001A or DS-011A or DS-016A
16	Combitorque	FV-060
17	External filter	FV-090
18	Pump (for Piezon 250)	BA-035B
N/A	Maintenance kit Piezon 150	FV-084
N/A	Maintenance kit Piezon 250	FV-085