OPERATION INSTRUCTIONS

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
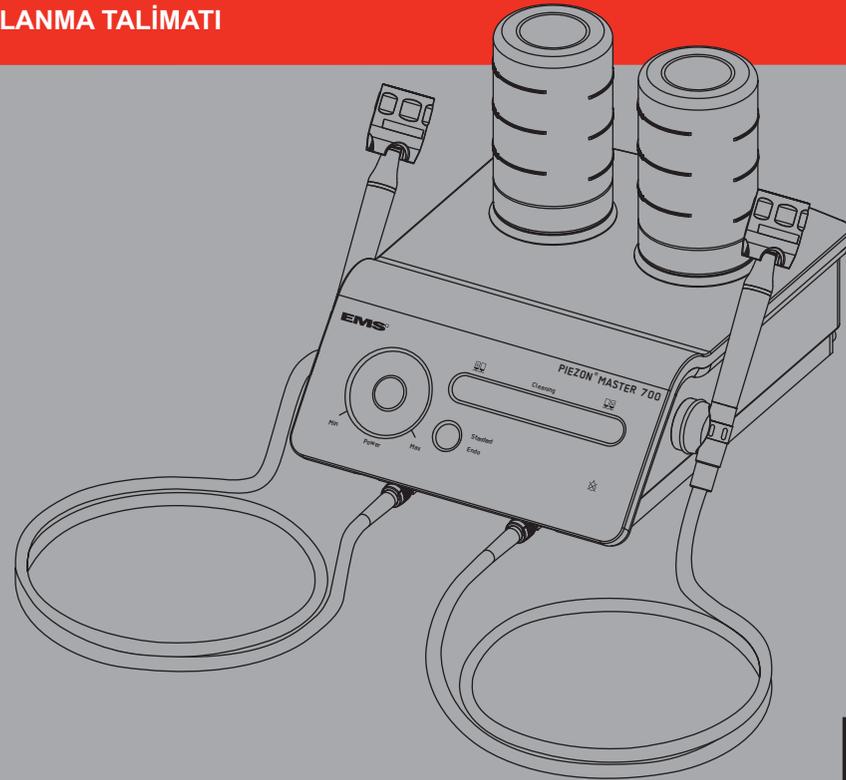
**EMS**   
ELECTRO MEDICAL SYSTEMS
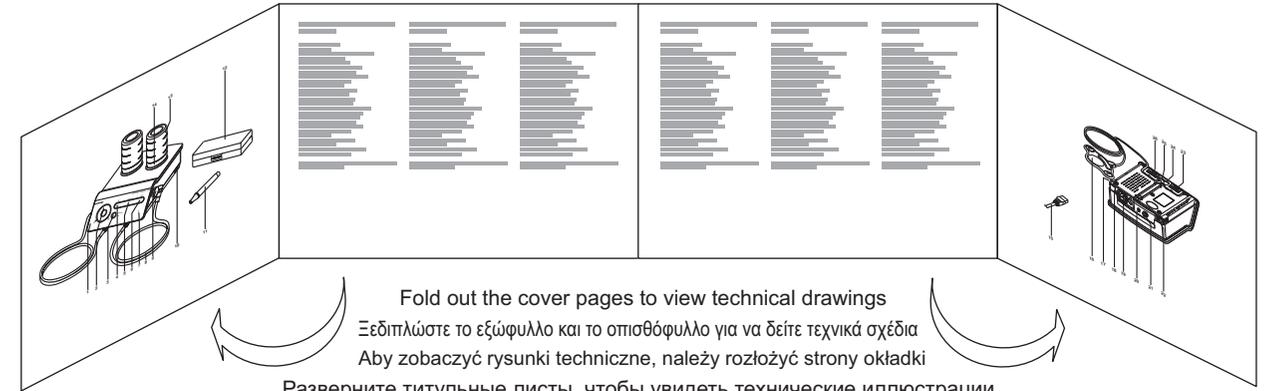
<https://stomshop.pro>

Fold out the cover pages to view technical drawings

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## TECHNICAL MODIFICATIONS

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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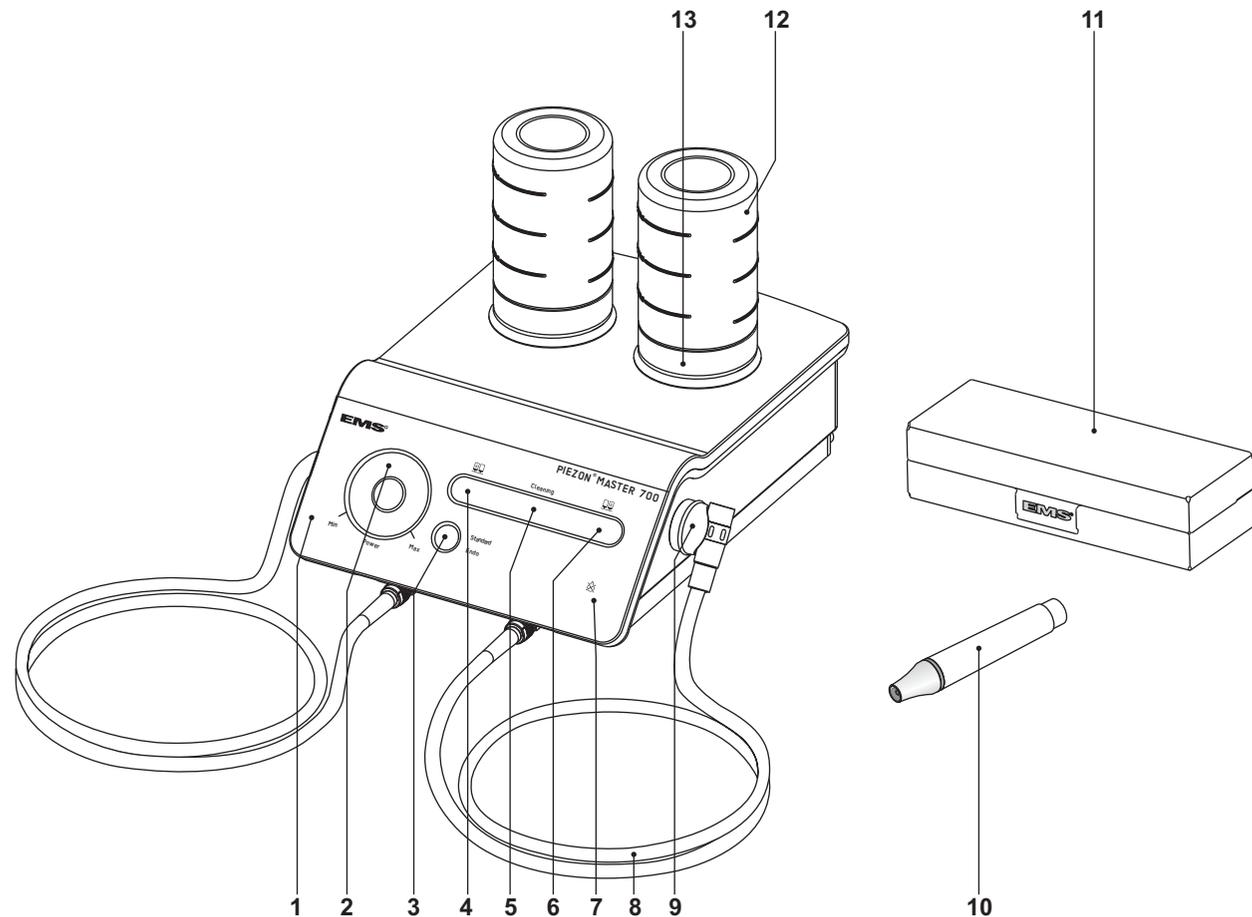
## COMPONENTS

- 1 Unit
- 2 Power setting
- 3 Working mode selection
- 4 Left bottle selection
- 5 "Cleaning" mode selection
- 6 Right bottle selection
- 7 "Dry Work" control lamp
- 8 Handpiece cord
- 9 Magnetic handpiece support
- 10 Handpiece
- 11 SystemBox Piezon®
- 12 Bottle
- 13 Bottle cap
- 14 Power cable
- 15 Foot pedal
- 16 Power switch
- 17 Power socket
- 18 Fuses
- 19 Unit pump cover
- 20 Screw of the unit pump cover
- 21 Connector for right handpiece cord
- 22 Connector for connecting a second EMS unit
- 23 Connector for the foot pedal
- 24 Connector for left handpiece cord

EMS supplies this unit with various accessories. The "Packing list" shows exactly what your unit includes.

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<https://stomshop.pro>

## DEAR CUSTOMERS,

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

### Indications

This product is designed for use in dentistry with EMS Piezon ultrasound instruments for the following indications:

### Scaling

- Removal of supragingival calculus
- Removal of stains

### Endo

- Preparation, cleaning and irrigation of root canals
- Retrograde preparation of root canals
- Condensing gutta-percha
- Removal of crowns, bridges and restorations

### Restorative

- Preparation of cavities
- Cementation of restorations
- Condensing of amalgams

### Perio

- Scaling and root planing
- Periodontal treatments

For further details on the various instruments and their scopes of application, please refer to the operating instructions supplied with your Piezon system or available from the distributor in your country.

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## ABOUT THIS MANUAL

These operating instructions are to ensure the correct installation and use of this product.

Please read these operating instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this document close at hand.

Instructions concerning the components of the system are provided separately in the respective instructions for use. Read the instruction manuals originally delivered with each of the components before using this product. Do not hesitate to contact your local EMS dealer for further information.

This product needs special precautions regarding electromagnetic compatibility and needs to be installed and put into service according to the "Electromagnetic compatibility" manual.

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To prevent possible patient or user injury or damage to the Piezon Master 700, please note the corresponding directives:

**Danger**

Risk of patient or user injury

**Caution**

Risk of damage to the device or environmental harm

**Please note**

Useful additional information and hints

**Prohibited****Authorized**

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please contact your approved EMS repair center or your dealer.

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EMS

## CONTENTS

Installation .....	2
Foot pedal .....	2
Handpiece cords and supports.....	4
Connecting to the mains power .....	6
Connecting to another EMS unit (option).....	8
Preparing for treatment.....	10
Installing the bottles .....	10
Solutions compatible with the circuit .....	12
Connecting the handpieces .....	14
Attaching the instruments .....	16
Operating the unit .....	18
Switching On .....	18
Working mode selection.....	20
"Dry Work" mode selection.....	24
Multifunction foot pedal.....	26
Piezon handpiece LED.....	28
Selecting the irrigation supply.....	30
Setting the ultrasound power .....	32
Setting the irrigation flow rate .....	34
End of the treatment .....	36
Instruments disassembly and cleaning .....	36
Cleaning and disinfecting the circuit.....	38
Removing the bottle .....	44
Cleaning, disinfecting and sterilizing .....	46
Cleaning the accessories.....	48
Maintenance.....	50
O-rings .....	50
Replacing the fuses.....	52
Replacing the pump of the unit .....	54
Safety precautions .....	56
Storing the product .....	60

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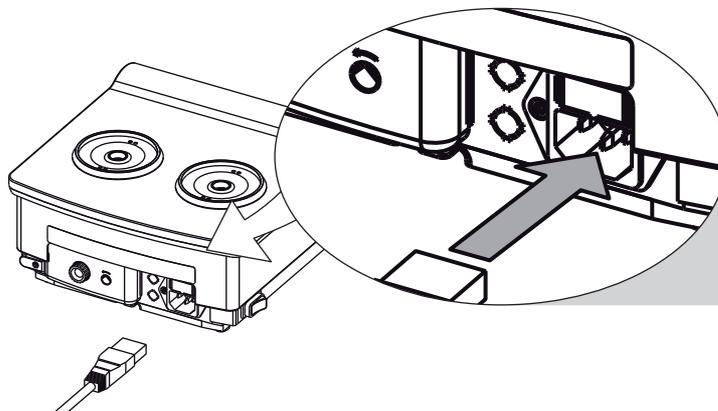
100-240 VAC 50-60Hz

 Always check that the unit is installed in a safe and stable position. An eventual fall may suddenly pull on the cord and the handpiece, hurting the patient and the user, and damaging the unit.

 Check that the rated voltage of the unit corresponds to the local line voltage to prevent damaging the unit.

Use the unit only in a FI protected mains supply (FI = Residual current protection).

 For USA or Canada: connect only to an Hospital Grade outlet.

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
## Connecting to another EMS unit (option)

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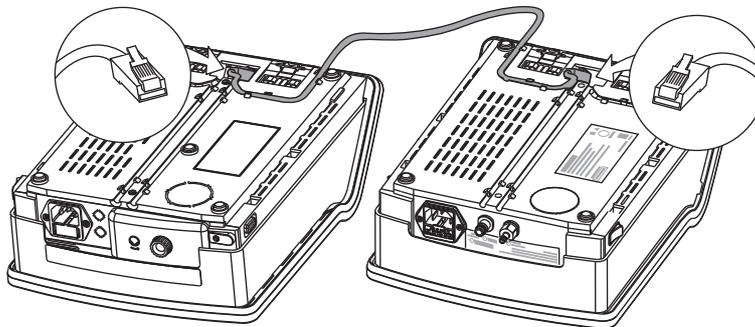The Piezon Master 700 can be connected to another compatible EMS device, so both devices can be operated using a single foot pedal. An optional link cable allows this connection. This option is only available with the "multi-function" pedal.

The pedal will trigger only the handpiece which is lifted from its support whatever the unit.

For additional details on device compatibility, please contact the distributor in your country.

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## Solutions compatible with the circuit

Please read the instructions for use of the disinfecting products.

Please refer to standards applicable in your country regarding the purchase and use of these products.

### Disinfecting solutions for the treatment

- Sodium hypochlorite ( $\text{NaOCl} \leq 3\%$ )
- Chlorhexidine ( $\leq 0.3\%$ )
- Physiological water ( $\text{NaCl} \leq 0.9\%$ )
- Hydrogen peroxide ( $\text{H}_2\text{O}_2 \leq 3\%$ )
- Carbanilides
- Povidone iodine
- Quaternary ammonium compounds
- Cetylpyridin chloride
- Citric acid
- Aromatic essence
- Ethanol

### Examples of common products

BacterX<sup>®</sup> pro / Triclosan / Betadine<sup>®</sup> / Listerine<sup>®</sup> / Alodont<sup>®</sup> / Meridol<sup>®</sup> / BreathRx<sup>®</sup> / CloSYS<sup>™</sup> / Dakin's solution / Scope<sup>®</sup> / Cepacol<sup>®</sup> /...

### Disinfecting solutions for washing the circuit

- EMS Ultra Clean ( $\text{H}_2\text{O}_2$  6%)
- Hydrogen peroxide ( $\text{H}_2\text{O}_2$ ) 5%
- Sterilox (hypochlorous acid)

### Incompatible mixing

Different disinfecting solutions can be used during treatment. Chemical reactions might occur between these solutions and damage the unit. Wash the liquid circuit before using a different solution (please refer to the "Cleaning mode" in the "End of treatment" chapter).

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
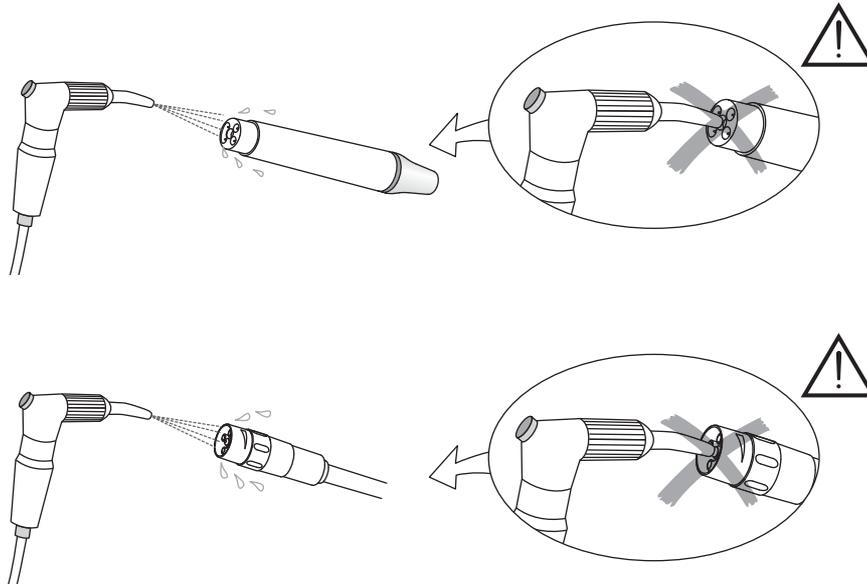
## Connecting the handpieces


⚠ Blow dry the connections to remove eventual presence of liquid to ensure a proper electrical contact. Never blow compressed air in the irrigation connections as this will irretrievably damage internal parts.

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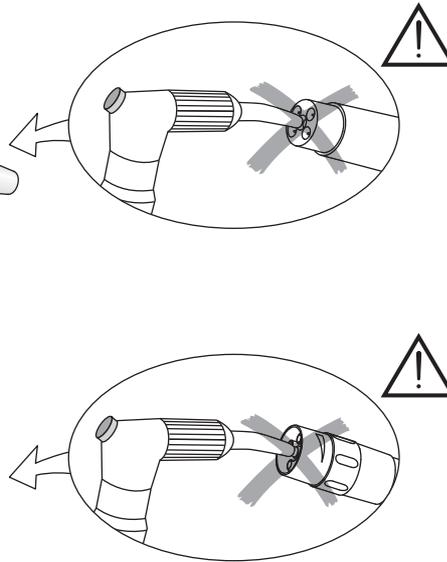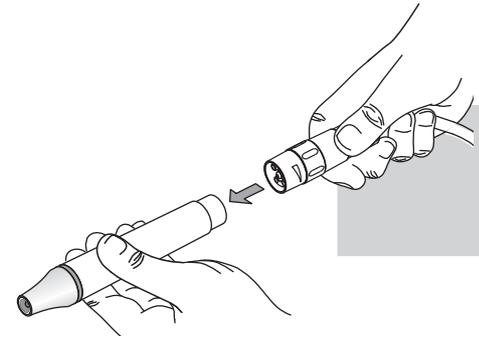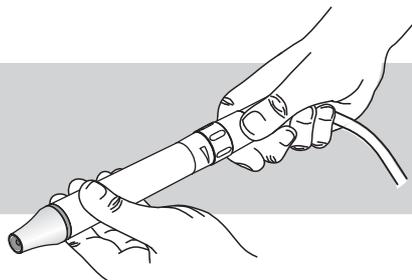
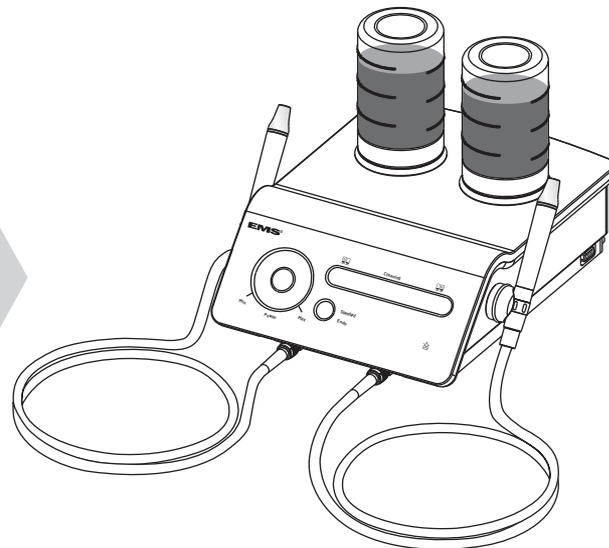
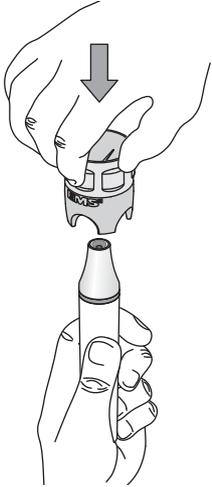
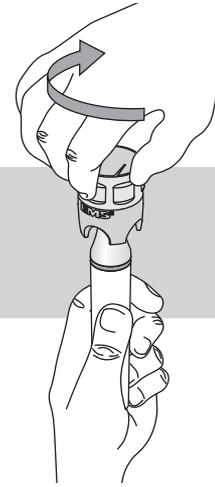
 Use only the CombiTorque to tighten the EMS instrument on the handpiece to the correct torque. Once the instrument is screwed all the way in, give an extra quarter of a turn to obtain the required torque.

 For further details on the use of the instruments, please refer to the operating instructions of the Piezon systems.

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
 The Piezon Power LED handpiece may not be available in all markets.

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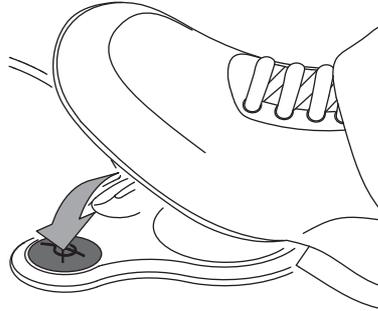## Wybór trybu "Dry Work"

Press the foot pedal on the corresponding symbol to activate the "Dry work" mode

Never work dry to avoid heat damage on the tissues unless the instrument is intended specifically for this purpose. The tip of the instrument immediately heats up when used without irrigation.

Please refer to the operating instructions of the Piezon systems for further information concerning instruments usable in "Dry work" mode.

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
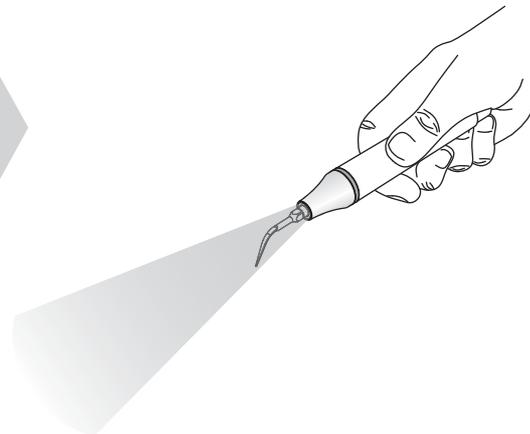
 The Piezon handpiece LED lights up when the foot pedal is activated. The light stays on for about 20 seconds after the foot pedal is released.

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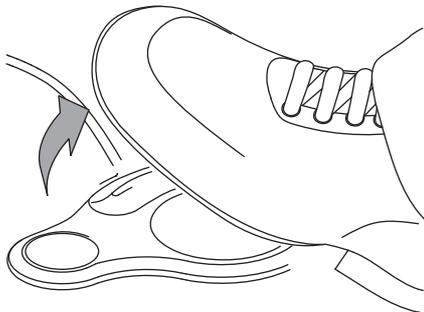
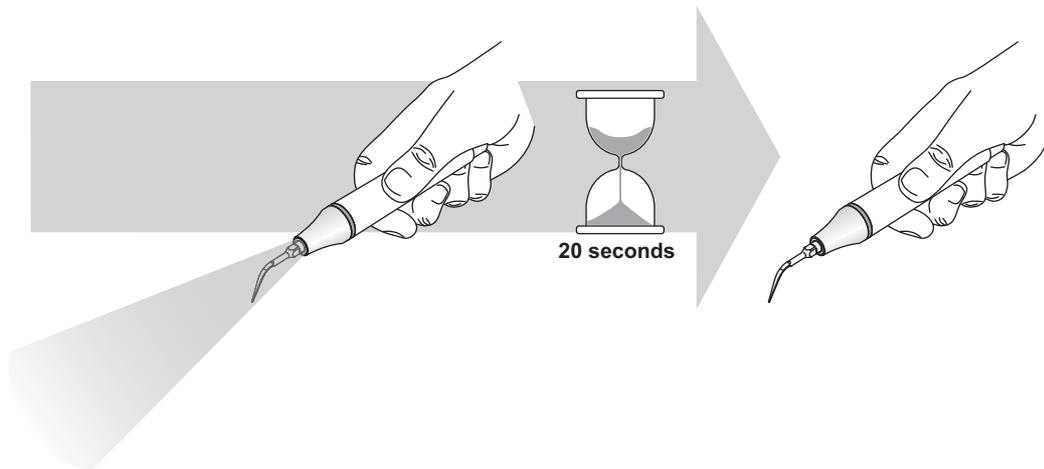
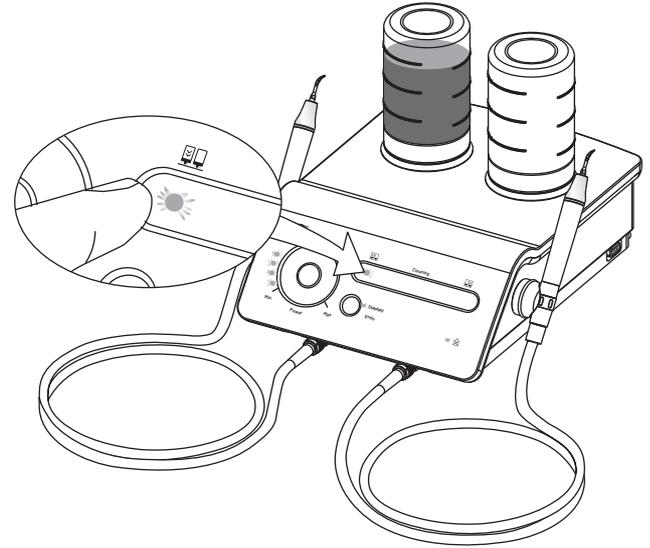
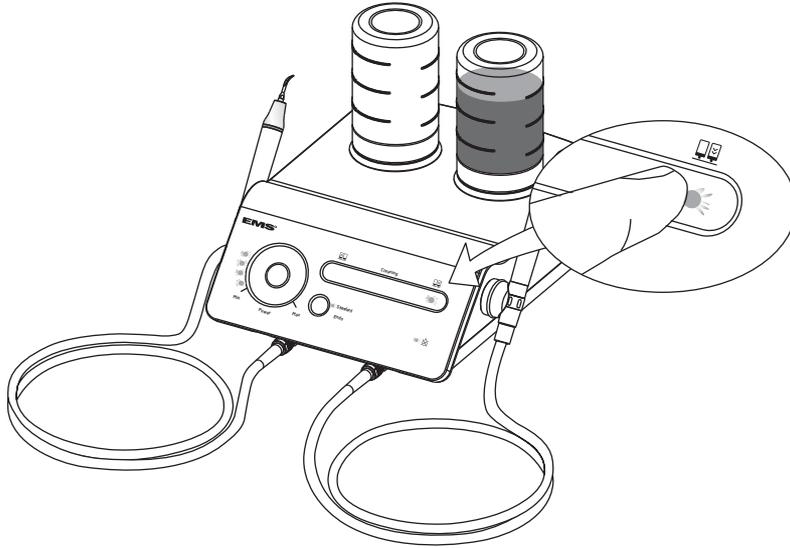
 Constantly check the presence of liquid in the bottles during treatment.

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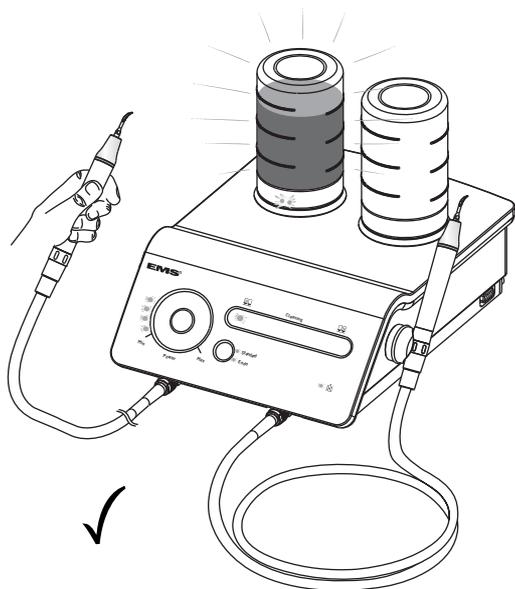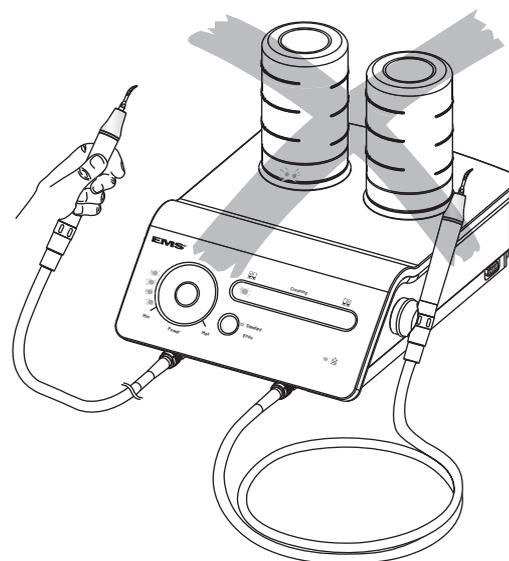
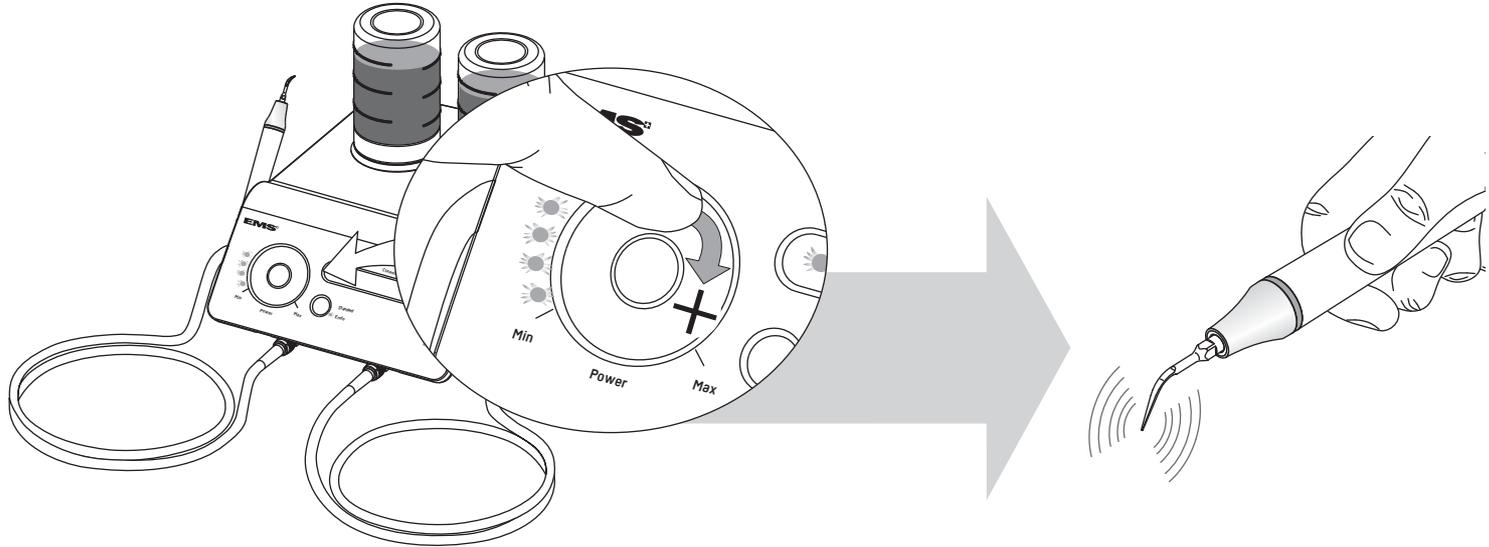
The settings can be obtained either by sliding or by placing the finger on the sensitive zone. To place the finger directly at the desired location is a quick setting alternative.

Optimal setting of the ultrasound power for each instrument are indicated in the operating instructions for the Piezon systems.

The use of a sterile on the unit may affect the the settings.

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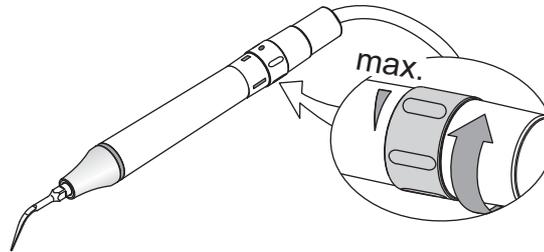
Optimal settings for the liquid flow rate for each instrument are indicated in the operating instructions of the Piezon systems

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

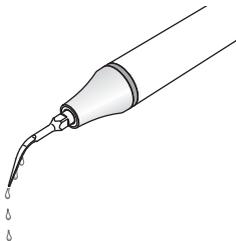
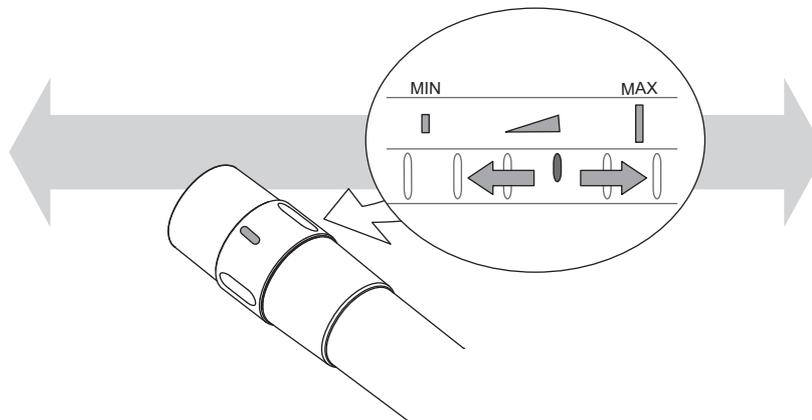
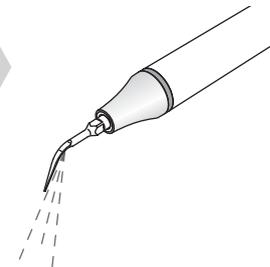
## END OF THE TREATMENT
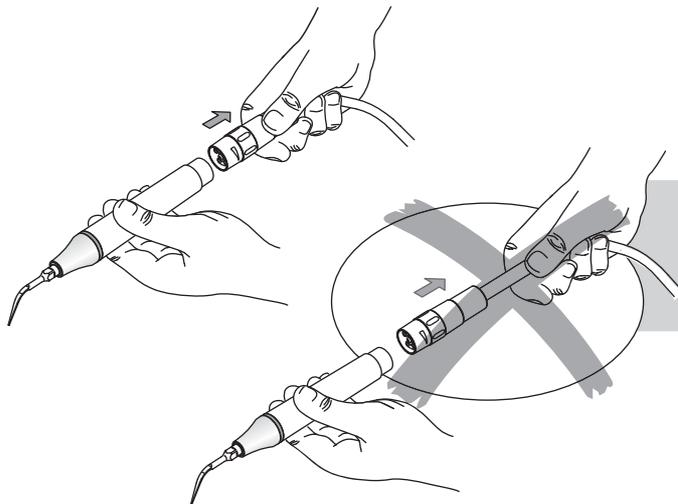
### Instruments disassembly and cleaning

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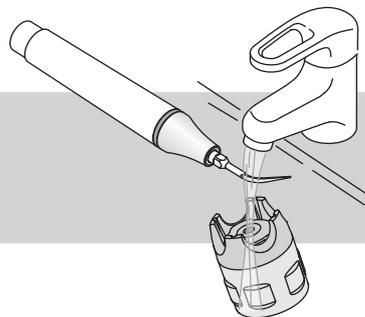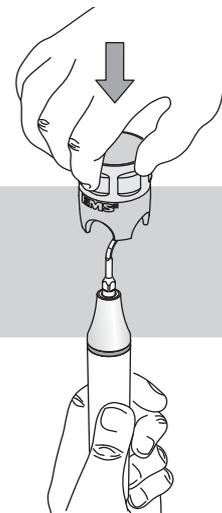
 Clean the instrument with running water before disinfection.

 Please conform to the recommendations of the "Reprocessing Instructions" manual delivered with your product regarding the procedures of cleaning, disinfecting and sterilizing of the components.

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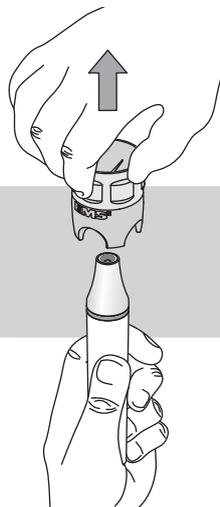
## Cleaning and disinfecting the circuit

### "Cleaning" mode

 It is recommended to proceed daily to the washing and the disinfecting of the liquid circuit of the unit.

 The "Cleaning" mode allows for the washing and the disinfecting of the liquid circuit in order to reduce the crystalline deposits and the number of bacteria in the liquid circuit. A rinsing of the circuit is then required to avoid a possible reaction caused by the ageing of the solution or by an unwanted mixture of liquids.

 All the bottles and handpiece cords used during treatment must be present during the washing and the disinfecting of the liquid circuit because the process is simultaneous for all these elements. Remove the handpieces.

### Circuit washing cycle:

- Prepare the bottle(s) with distilled or demineralized water,
- Press the "Cleaning" mode selector,
- Select the bottle(s) to use,
- Lift the connector(s) and put it (them) above a sink,
- Press the foot pedal to start the program. Three "beeps" and the blinking of the control lamp of the "Cleaning" mode confirms the end of the cycle.

### Disinfecting cycle:

- Prepare the bottle(s) with a compatible disinfecting solution (refer to the list of compatible disinfectants at the beginning of this manual),
- Select the bottle(s) to use,

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- Lift the connector(s) and put it (them) above a sink,
- Press the foot pedal to start the program. Three "beeps" signal and the blinking of the control lamp of the "Cleaning" mode confirms the end of the cycle,
- Then allow the disinfecting solution to act for 5 minutes without intervene with the unit.

#### Rinsing cycle:

- Prepare the bottle(s) with the sterile distilled water,
- Select the bottle(s) to use,
- Lift the connector(s) and put it (them) above a sink,
- Press the foot pedal to start the program. Three "beeps" signal and the blinking of the control lamp of the "Cleaning" mode confirms the end of the cycle,

Press the "Cleaning" mode selector to exit this mode (this cycle must be then restarted).

(Note: it is possible at any time to press the foot pedal to stop the program in progress.)

The informative durations of the "Cleaning" mode cycles are defined as follows (Note: one led corresponds to 10 seconds):

	1 bottle	2 bottles
1 cord	30 seconds 50 ml	1 minute 2 x 50 ml
2 cords	1 minute 100 ml	2 minutes 2 x 100 ml

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


## CLEANING, DISINFECTING AND STERILIZING

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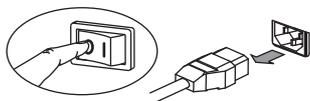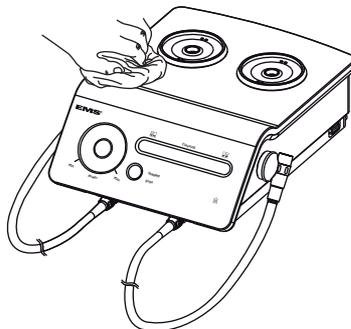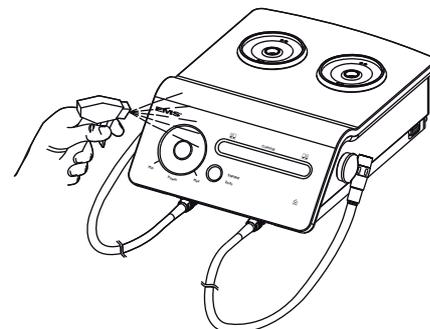 Please conform to the recommendations of the Reprocessing Instructions manual delivered with your product regarding procedure of cleaning, disinfecting, sterilizing and packing of the components.

 Follow present-day regulations enforced in the country about reprocessing.

 Clean the unit only with an alcohol-based, commercially available (ethanol, isopropanol), colorless disinfectant. The use of scouring powder or an abrasive sponge will damage its surface.

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## SAFETY PRECAUTIONS

EMS and the distributor of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 **Contraindication:** ultrasonic oscillations may prevent cardiac pace makers and defibrillators from functioning properly. Therefore, we recommend that patients with a cardiac pace maker or a defibrillator should not be treated with this product.

 **Use for the intended purpose only.** Before using the product, make sure the operating instructions have been carefully studied. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged.

 **Risk of explosion:** do not use this product in the presence of inflammable anaesthetics or gases.

 This product must be used only by trained and qualified personnel.

 Check before each treatment the contents of the bottle as well as the liquid present in the circuit.

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 Clean, disinfect and sterilize the different pieces and accessories of this product before each use. Please refer to the information provided in the operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

 Always examine the product for damage before starting the treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved EMS repair center.

 This product has been investigated with regard to safety from electrical shock and fire hazard. CSA (Canadian Standard Association) has not investigated physiological effects.

 Certain types of mobile telecommunication equipment could potentially interfere with this product. The separation distances recommended in the note "Electromagnetic compatibility" manual must be taken into account.

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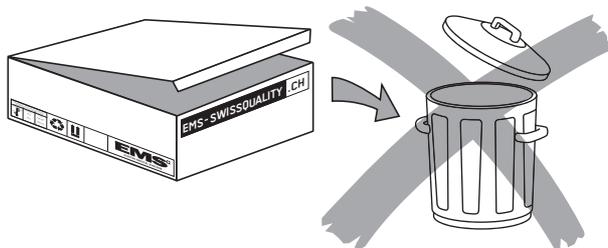
 Keep the original packaging until the product is to be disposed of permanently. It can be used for shipping or storing at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the "Cleaning, disinfecting and sterilizing" chapter,
- Proceed to a "Cleaning" cycle without bottles to empty the circuits,
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data" chapter.

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## PRODUCT DISPOSAL

The product must not be discarded in domestic household waste.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

Waste Electrical and Electronic Equipment belonging to customers located in the European Union may be shipped to EMS for recycling in accordance to the WEEE regulations. The costs of recycling, exclusive of shipping fees, are covered by EMS.

## WARRANTY

Damages due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

 The warranty of your product will be cancelled if you try to open it.

## ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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Should your product need additional servicing or repairs, please send it to your dealer or to an authorized EMS Repair Center.

In the case of non-authorized repairs or damages due to non-adherence to the operating instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship the product in the original packaging. It protects the product against damage during shipment.

Please clean, disinfect and sterilize the product and all accessories as described in the operating instructions before dispatching them.

When sending the product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing.

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## TECHNICAL DATA

### DESCRIPTION

Manufacturer	EMS ELECTRO MEDICAL SYSTEMS SA, CH-1260 Nyon, Switzerland
Model	Piezon® Master 700
Classification EN 60601-1	- Class I - Applied part, Type BF - IP 20, unit - IP X1, foot pedal
Classification 93 / 42 EWG	Class IIa
Mode with irrigation: without irrigation:	Continuous operation Duty cycle 10% during 10 min. maximum
Supply voltage	100-240 VAC / 50-60 Hz
Power consumption	80 VA
Fuses	T 1.6 A 250 VAC (Ø5 X 20mm, time lag)
Ultrasound specifications Maximum output: ..Frequency range:	12 Watts 24-32 kHz
Weight	Approx. 2.4 kg
Dimensions in mm (HxWxD)	225 x 280 x 295
Operating conditions	+10 °C to +40 °C 30% to 75% relative humidity
Storage and transport conditions	-10 °C to +40 °C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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<https://stomshop.pro>

# TROUBLESHOOTING

Symptom	Solution
Control lamps do not turn on when the unit is switched on	<ul style="list-style-type: none"><li>• Check the connections of the power cable of the unit</li><li>• Check the power supply of the dental office</li><li>• Check the fuses and replace them only by the type of fuse indicated on the back of the unit</li><li>• Send the unit to an approved EMS repair center</li></ul>
No liquid flow when the foot pedal is activated	<ul style="list-style-type: none"><li>• Turn the liquid flow rate ring to the maximum</li><li>• Make sure that the bottle is properly installed</li><li>• Check the connections between the unit, the handpiece cord and the handpiece</li><li>• Check the connection of the foot pedal</li><li>• Activate the "Cleaning" mode, disconnect the handpiece from the handpiece cord and check that liquid exits the cord; if it is the case, the handpiece is plugged, send the handpiece to an approved EMS repair center</li><li>• Unscrew the pump cover and check that the peristaltic pump functions</li><li>• Send the unit to an approved EMS repair center</li></ul>
No ultrasound oscillations or insufficient power	<ul style="list-style-type: none"><li>• Make sure that the instrument is correctly tight</li><li>• Check the wear of the instrument and replace it if necessary</li><li>• Check the connections between the unit, the handpiece cord and the handpiece</li><li>• Dry the connectors of the handpiece cord and of the handpiece</li><li>• Check the connection of the foot pedal</li><li>• Check the settings and the working mode</li><li>• Check that when the handpiece is lifted from its support, the control lamps of the bottle holder are turned on; if it is not the case, send the handpiece to an approved EMS repair center</li><li>• Replace the handpiece cord and/or the handpiece</li><li>• When one of the two handpieces is used, check that the other one is correctly seated on its support</li><li>• Send the unit to an approved EMS repair center</li></ul>
Water leak between the handpiece cord and the handpiece	<ul style="list-style-type: none"><li>• Check the O-ring of the handpiece cord and replace if necessary</li><li>• Send the handpiece cord to an approved EMS repair center</li><li>• Send the unit to an approved EMS repair center</li></ul>
Water leak at the bottle or at the peristaltic pump	<ul style="list-style-type: none"><li>• Check the O-ring of the bottle cap and replace if necessary</li><li>• Replace the bottle</li><li>• Replace the peristaltic pump</li><li>• Send the unit to an approved EMS repair center</li></ul>
Insufficient lighting of the handpiece	<ul style="list-style-type: none"><li>• Replace the light guide</li><li>• Test with another handpiece</li></ul>

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EMS Corporation