SybronEndo

elementsTMfree

OBTURATION SYSTEM

Instructions for Use

Manufactured for SybronEndo
1332 S. Lone Hill Avenue, Glendora, CA 91740 U.S.A.
714.516.7979 | SybronEndo.com


MADE IN KOREA

 META BIOMED CO., LTD.
270, Osongsaengmyeong1-ro, Osong-eup,
Heungdeok-gu, Cheongju-si, Chungcheongbuk-do,
Korea

EC REP
Technical and General
2 Albion Pl. London W6 0QT,
United Kingdom

CE
0120

Posted, 17 Mar 2015, G-ENDO DWG-REL
077-3246 Rev. B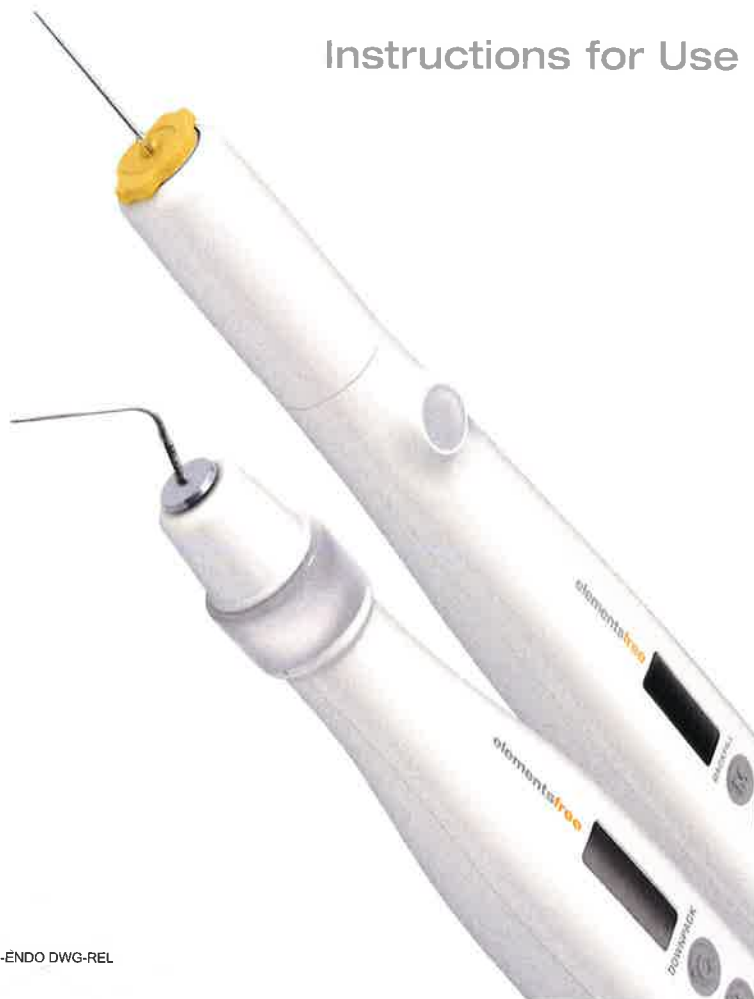

CONTENTS



English	2
Français	8
Deutsch	14
Español	20
Italiano	26
Português	32
Nederland	38
Dansk	44
Norsk	50
Suomi	56
Svenska	62
Česky	68
Magyar	74
Polski	80
Русский	86
Hrvatski	92
Serbian	98
Slovenčina	104
Eesti	110
Lietuvių	116
Română	122
Türkçe	128
Ελληνικά	134
繁體中文	140
한국어	146
日本語	152
Arabic	158

elementsTMfree

OBTURATION SYSTEM

The cordless Elements™ Free obturation system combines a breakthrough Downpack heat source with a world-class Backfill extruder. The Downpack device provides instantaneous heating and cooling of the heat plugger with precisely controlled temperature and timing—making it perfect for single-motion downpack obturation and creating post space. Utilizing single-use gutta percha needle cartridges, the ergonomic Backfill device has a motorized extruder system for precise temperature and speed control for a 3-dimensional fill when backfilling canals.

CONTENTS OVERVIEW

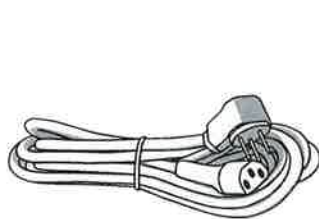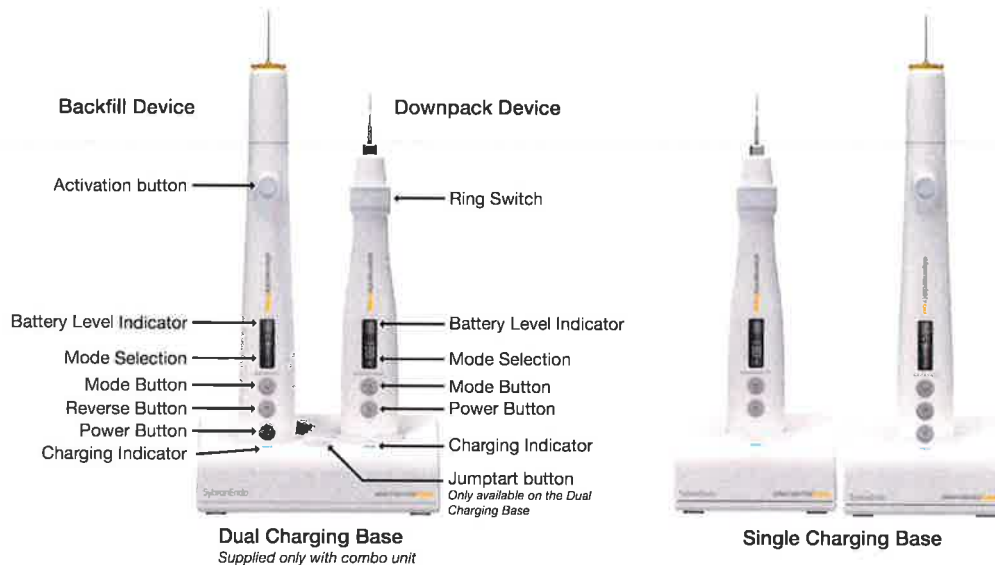Power Cord

Transformer


Sleeve Covers (5 PCS)

Heated Plugger
Supplied only with Downpack and combo unit

Needle Bender
Supplied only with Backfill and combo unit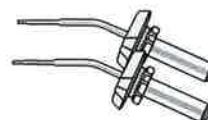

Backfill Cartridges (2 PCS)
Supplied only with Backfill and combo unit

WARNING

The System B tips and Backfill cartridges are designed to reach high temperatures. Inadvertent contact with patient and operator must be avoided to prevent accidental burns.

Handle empty extruder cartridge by plastic locknut only. Other portions could be hot and could result in serious burns.

Extruder cartridges are for single-patient use.

This unit should NOT be used on a patient with a pacemaker.

Only use the specified SybronEndo AC power supply and power cords.

Use of other accessories that are not authorized for use in connection with this device may cause malfunction and compromise patient safety.

No modification of this equipment is allowed.

CAUTION

This device has been investigated with regard to safety from electrical shock and fire hazard as well as electromagnetic compatibility (EMC). The device has not been investigated for other physiological effects. Please contact SybronEndo if you have any further questions regarding electrical safety or EMC.

For use by qualified and trained personnel only.

This device has been tested and found to comply with EMC limits for the Medical Device Directive 93/42/EEC. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. The device generates radio frequency energy and, if not installed and used in accordance with these instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. If this device does cause harmful interference with other devices, which can be determined by turning the device off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving device.
- Increase the separation between devices.
- Connect the device into an outlet on a circuit different from that to which the other device(s) is connected.
- Consult the manufacturer for help.

To reduce the risk of electrical shock, do not remove cover. Refer servicing to qualified service personnel.

U.S. federal law restricts the sale of this device by or on the order of a healthcare professional.

Remove main plug from wall socket to isolate unit from main power.

Position device such that it can be easily unplugged from the wall.

Symbols

	Serial Number		European Representative
	Manufacturer		Corresponds to MDD 93/42 EEC Including EN 60601-1 and EN 60601-2
	Manufacturing Date		CSA mark with "C/US" indicator for certified products
	This way up		Caution/Warning
	Fragile, handle with care		Consult accompanying documents
	Keep away from rain		Type BF Patient Applied Part
	Temperature limits for storage and transportation		Class II Medical Electrical Equipment
	Humidity Limitation		Do no autoclave
	Pressure Limitation		Part Number
	Do not throw into trash		Caution: Hot!

GETTING STARTED

1. INITIAL SET-UP

- Remove the device(s), Charging Base, transformer and wall cords from their packaging.
- Connect the female end of the power cord into the transformer. Then plug the ends into the wall socket and the Charging Base.
- On the bottom of the device(s), make sure the switch is set to the "Unlocked" position.
- Place the handpiece(s) into the Charging Base, and confirm that they are firmly seated.

2. CHARGING THE UNITS

- Once the handpieces are firmly seated, the indication lights on the Charging Base will be either orange or blue.
- The light will be orange when the battery is charging and will turn blue when the battery is fully charged.

ORANGE

Orange means the unit is charging

BLUE

Blue means the unit is fully charged

3. POWERING ON THE UNITS

- To power on the device(s), press the Power button "P".
- If you have the Dual Charging Base, and both units are seated firmly in the base, press the Jumpstart button "J" to power on both units at once.

4. SET TEMPERATURE MODE

- Devices come with the three preset temperature modes. The following table shows their function and default temperature settings:

Setting	Description	Default Temp.
GUTTA PERCHA	For use with Gutta Percha	200°C
SYNTHETIC	For use with synthetic filling materials	150°C
CUSTOM	For use as an additional preset.	200°C

- Press the Mode button to toggle to the appropriate preset.

For instructions on changing the default temperature for a preset, see "Changing Temperature Presets" in the ADVANCED SETTINGS section.

5. POWERING OFF DEVICE(S)

The unit will automatically shut down after approximately 5 minutes. To manually turn off the device, hold down the Power button for approximately 2 seconds. The unit will generate an audio signal to indicate it is powering down. Upon any power off of the Backfill, the motor will automatically retract for a few seconds to relieve the pressure on the cartridge.

Hold 2 Seconds

DOWNPACK

1. INSTALLING THE TIP

- Index the hex nut to slide into one of six positions available.
- Install the tip until it is fully seated.

	CAUTION Older System B tips without this hex nut or other manufacturer's tips may not operate correctly and may damage the unit.	
--	--	--

2. ACTIVATING THE DOWNPACK

Activate the handpiece by depressing and holding the Ring switch. The plugger will heat instantly and a light under the Ring switch will illuminate.

CAUTION:
Tip will be hot

	NOTE - To avoid overheating, the plugger will only remain heated up to a maximum of approximately 4 seconds regardless of how long the button is depressed. - Indicated temperature is the temperature the tip of the plugger reaches after activation in air. As the tip contacts other materials, its temperature may decrease depending on the nature of the contact.
--	---

	CAUTION The heat pluggers must be autoclaved between every patient to prevent cross-contamination. See the instructions in the CLEANING AND STERILIZATION section.
--	--

BACKFILL

1. INSERTING CARTRIDGE

- Insert the large end of the Elements Cartridge into the handpiece and rotate a quarter turn clockwise (CW) until it engages.

CAUTION: HOT!

When heated at the peak temperature for an extended period of time, plastic housing surface may exceed 85 C. Accordingly, it is recommended to turn the device off after 15 minutes of use and allow it to cool.

CAUTION

Elements Cartridges are SINGLE USE ONLY and should only be used on one patient. Re-use may lead to cross-contamination between patients.

2. HOW TO TELL WHEN THE BACKFILL IS READY FOR USE

- Once powered on, the Backfill unit will automatically begin to heat. The illuminated Activation button will indicate whether or not the device is ready for use.

Flashing Light:
Device is still heating and is NOT ready for use.

Steady Light:
Device is fully heated and ready for use.

Press & hold

Manual Run Mode:
a. Hold down the activation button to engage motor.
b. Release activation button to stop.

"Click" "Click"

Continuous Run Mode (CR):
a. Double-Click to engage CR mode.
b. Press button once to stop.

NOTE

Before placing the cartridge needle in the canal, activate the motor until material extrudes out of the needle. If the motor is stopped before the cartridge is empty, the plunger will retract slightly to prevent excess material from discharging from the needle. Once the plunger reaches the end of the stroke, the cartridge is empty and the plunger will automatically retract.

4. RETRACTING THE PLUNGER

- To retract the plunger before it reaches the end of its stroke, press the Reverse button. This will cause the plunger to fully retract to its "home" position.

5. REPLACING THE CARTRIDGE

- To replace a cartridge before it is completely empty, first ensure that the unit is fully heated, then press the Reverse button to retract the plunger.
- Remove the cartridge by turning the nut 90° counterclockwise (CCW), then pull the cartridge out of the handpiece and dispose of properly.

CAUTION: HOT!

Handle empty cartridges by the plastic locknut only. Other portions could be hot and could result in a serious burn.

ADVANCED SETTINGS

1. CHANGING TEMPERATURE PRESETS

The unit has three different preset temperature modes. (See the table in Step 4 of the GETTING STARTED section)

- Press the Mode button to toggle to the temperature setting you would like to adjust.
- Hold down the Mode button for approximately 2 seconds to enter Temperature Control Mode.
- Press the Mode button repeatedly to change temperature.
- Once desired temperature is reached, hold down the Mode button (approx. 2 seconds) again to exit Temperature Control Mode.

NOTE

- When the temperature setting on the GUTTA PERCHA or SYNTHETIC modes is changed from the default setting, an asterisk "*" will appear next to the temperature.
- On the Backfill device, only the CUSTOM temperature setting can be adjusted.

CAUTION (BACKFILL ONLY)

Keep temperature setting above 150° C. Extruding material at lower temperatures will create more stress on the motor, and may shorten the device's life.

2. ADJUSTING THE VOLUME

- Change the volume level by simultaneously holding down the Mode and Power buttons for approximately 2 seconds. This will open volume control mode.
- Press the Mode button repeatedly to change the volume level.
- Once you have set your desired level, exit volume control mode by repeating the first step (a) above.

Hold 2 Seconds

MAINTENANCE

1. CHARGING THE DEVICE(S)

The handpieces should be placed onto the Charging Base during and between uses. This will ensure that the batteries are maximally charged.

2. REPLACING THE BATTERY

Contact Customer Support if the battery appears to be malfunctioning. To replace battery:

- Remove screw on battery cover and remove battery.
- Install new battery in same orientation.
- Dispose of used battery per local regulations.

CAUTION

Only use SybronEndo batteries designated for this system. Use of other batteries may damage the device(s) or cause a malfunction.

3. POWER LOCK FEATURE

The device(s) can be locked in a power-off mode by sliding the switch on the base from the "unlock" position to the "locked" position (See Step 1 in GETTING STARTED).

CLEANING AND STERILIZATION

1. CLEANING THE DEVICE(S) AND CHARGING BASE

Clean all the surfaces with a cloth lightly moistened with isopropyl alcohol. To prevent cross-contamination, single-use barriers can be used on the hand pieces. SybronEndo recommends TotalCare's Pinnacle Syringe Sleeves (REF 3760).

CAUTION

- The electronic components should not be autoclaved as it will damage the circuitry.
- Do not spray devices with any liquids as it may damage the circuitry.
- Do not allow liquid to collect in the Charger base or come in contact with the connectors it may damage the circuitry.

2. CLEANING THE HEATED PLUGGERS

Clean with water and mild, nonabrasive detergent such as dishwashing liquid. Dry thoroughly. The heat pluggers must be autoclaved between every use. To avoid damaging the pluggers, place them in the System B tip stand before autoclaving. Operate sterilization cycle at 250°F (121°C) for 30 minutes or 275°F (134°C) for 6 minutes.

CAUTION

Do not place pluggers into an ultrasonic bath as it may damage the unit.

3. CLEANING THE CARTRIDGE NEEDLE BENDER

The cartridge needle bender should be wiped down with a cloth moistened with isopropyl alcohol. Sterilize by steam autoclave at 250°F (121°C) for 30 minutes or 275°F (134°C) for 6 minutes.

TROUBLESHOOTING GUIDE AND TECHNICAL SUPPORT

If a problem occurs with your device, refer to the guide below. If this guide does not resolve the problem, contact Customer Care at 800-346-3636 or 714-516-7979 (Available from 6:00am to 5:00pm PST). Outside of these hours, email us at customer-care@sybrondental.com. Alternatively, contact your dealer or SybronEndo sales representative.

Problem	Possible Causes	Solution
Device(s) do not turn on	Lock switch on base of unit in "locked position"	Slide Power Lock switch to "unlocked" position. (See Step 1 in GETTING STARTED section)
	Battery out of charge	Place the devices on the Charging Base.
Charging base light(s) do not turn on	Base is not getting power	Ensure wall outlet has power. Check and adjust all connections: wall-outlet plug, power cord plug into transformer and the small plug into the Charging Base.
	Device(s) may not be fully seated	Gently wiggle the devices to ensure proper seat.
Heat plugger on Downpack unit is not heating	Heat plugger not seated into Downpack unit	See Step 1 in DOWNPACK section.
	Heat plugger tip is burned out	Replace tip.
	Battery is out of charge	Check battery level on display. If battery level indicator is low, place the device on the Charging Base.
Device(s) not heating	Battery is out of charge	Check battery level on display. If battery level indicator is low, place the device on the Charging Base.
Filling material/gutta percha do not extrude	Motor is stuck or plugged due to gutta percha in the unit.	Let the device heat up. Press Reverse button. DO NOT disassemble the device(s).
	Temperature not set correctly	Adjust temperature setting. See Step 4 in the GETTING STARTED section; See "Changing Temperature Presets" in the ADVANCED SETTINGS section.
	Cartridge is empty or defective	Replace cartridge.
Unable to remove used cartridge from Backfill device	Plunger inside the Backfill device is stuck.	Heat up the device. Press Reverse button. DO NOT disassemble the device.
The light on charger fails to turn from orange to blue	Battery may be near end-of-life	Wait at least 4 hours. If unit fails to reach full charge, replace battery. Only use SybronEndo battery.
Battery becomes exhausted quickly	Battery may be near end-of-life	Replace battery. Only use SybronEndo battery. Note: On the Backfill device, battery has just enough power for one procedure, roughly 15 minutes and one cartridge. It is recommended to place the device on the Charging Base between uses.

PART NUMBERS

PARTS

Elements Free Obturation System	973-0500-TYPEX*
Downpack Unit	973-0502-TYPEX*
Backfill Unit	973-0504-TYPEX*
Dual Charger	973-0510
Single Charger	973-0512
Transformer	973-0515
Power Cord	973-0516-TYPEX*
Battery	973-0520

Heated Pluggers

Medium Large (ML)	952-0007
Medium (M)	952-0006
Fine Medium (FM)	952-0005
Fine (F)	952-0004
Extra-Fine (XF)	952-0031

Elements Cartridges

Gutta Percha, Medium Body, 23 Gauge	972-1002
Gutta Percha, Heavy Body, 23 Gauge	972-1005
Gutta Percha, Light Body, 25 Gauge	972-1003

SPECIFICATIONS

- Battery: 3.7 V, 1700 mAh Li-Ion, 6.3 Wh
- Adapter: Input: 100-240V~, 50/60 HZ, Output 9V = 2A
- Downpack: ~155mm x 28mm x 28mm (L x W x H), ~98 g
- Backfill: ~ 195mm x 28mm x 28 mm (L x W x H), ~155 g
- Device: IEC 601-1 Class II, Type BF Device
- Mode of Operation: Continuous Operation

*X designates plug type.

Type B		
Type F		
Type G		
Type I		
Type N		

CONDITIONS

Storage and Transport Conditions:	Conditions for use:
 -10°C (14°F) to 40°C (104°F) Temperature	 10°C (50°F) to 35°C (85°F) Temperature
 10% to 90% Relative Humidity	 30% to 75% Relative Humidity
 500 hPa to 1400 hPa Atmospheric Pressure	 697 hPa to 1013 hPa Atmospheric Pressure
	0-10,000 ft. Altitude Limitation

WARRANTY

SybronEndo warrants the system (excluding batteries) to be free from defects in materials or workmanship for period of 2 years from the original date of purchase. The batteries are warranted for a period of 6 months from the original date of purchase. If the system shows any defect within the warranty period that are not excluded from this warranty, SybronEndo shall, at its sole discretion, either replace or repair the device using suitable new or reconditioned parts. In the case other parts are used which constitutes an improvement, SybronEndo may, at its discretion, charge the customer for the additional cost of these parts. If the warranty claim provides to be justified, the product will be returned to the user freight prepaid. Warranty claims other than those indicated herein, are expressly excluded.

EXCLUSIONS

Damage and defects caused by the following conditions are not covered by the warranty:

- Improper handling/disassembly/modifying, neglect, or failure to operate the unit in compliance with the instructions given in this manual.
- Force majeure or any other condition that is beyond the control of SybronEndo.
- Damage caused by customer misuse or uses other than those specified.

DISCLAIMER

For safety reasons, this product should be used with accessories manufactured and sold by SybronEndo. Any use of non-authorized accessories or not following any of the instructions for use is done so at the operator's risk and voids the warranty. SybronEndo does not assume any responsibility for incorrect diagnosis due to operator error or equipment malfunction.

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