







Finishing & Polishing Instruments

# Pro-Tech Point-EX


Pro-Tech Point-EX is a diamond point tools for ceramists. Each one is made by special materials and has a customized shape for curving or polishing. They will shorten your working time.



## Diamond Point

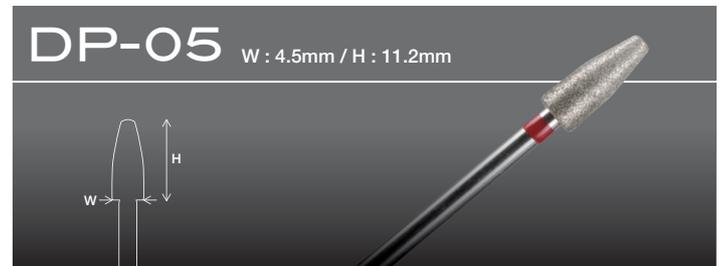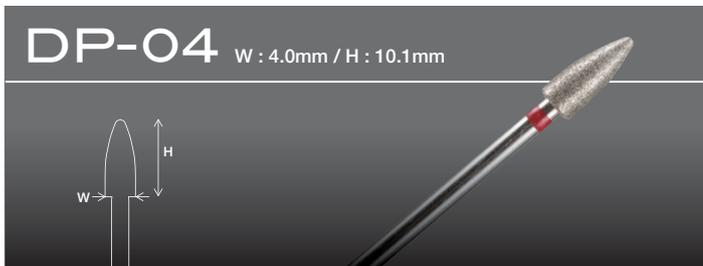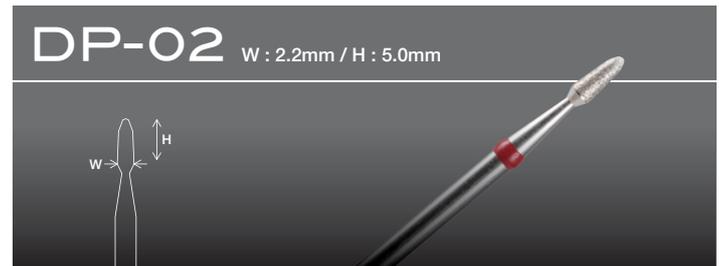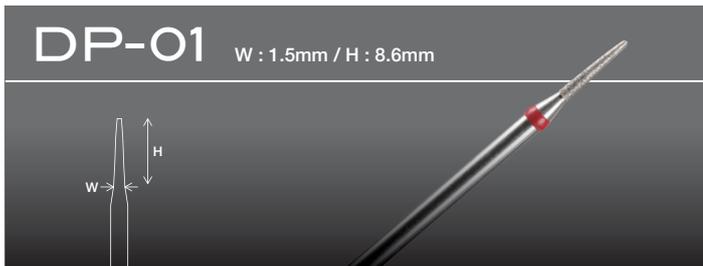

## Diamond Particle included Rubber Point

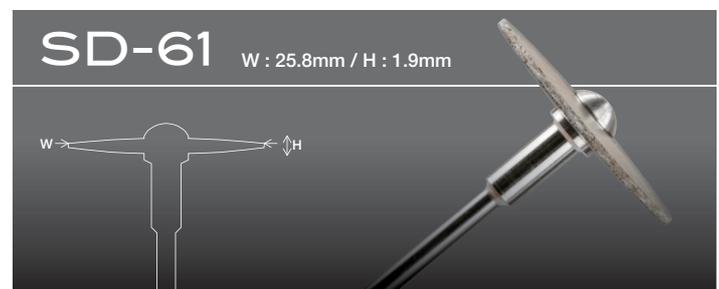

# Finishing & Polishing Instruments

## Pro-Tech Point-EX

Pro-Tech Point-EX is a diamond point tools for ceramists.

Each one is made by special materials and has a customized shape for curving or polishing.

They will shorten your working time.

SD-61



- Grinding zirconia or alumina framework

DP-01



- Adjusting the embrasures
- Create growth line

DP-02



- Surface textures
- Occlusal surface of molars
- Correction of pontic bottom

DP-04



- Surface textures
- Occlusal surface of molars
- Adjusting embrasures
- Correction of pontic bottom

DP-05



- Surface textures
- Occlusal Surface of molars
- Adjusting Contacts

DP-06



- Basic anatomy
- Surface textures
- Adjusting contacts

DP-07



- Occlusal Surface of molaars
- Adjusting angle of anteriors

SC-51



- Touching-up before glazing baking
- Correction of pontic bottom
- Correction of zirconia or alumina framework

### Pro-Tech Point-EX Assort Set



include : DP-01, DP-02, DP-04, DP-05,  
DP-06, DP-07, SD-61, SC-51 each  
1 piece

### Rotating Speed (Max.)

DP	15,000 r.p.m.
SC	20,000 r.p.m.
SD	15,000 r.p.m.



Use below max. allowable r.p.m. of each instrument to avoid a possibility of being injured due to breakage.

### Precautions

1. Be sure to sterilize a point for each use.
2. Before actual use, run up to high speed and check for wobbling.
3. Before using SD-61, be sure that the screw is tightened.
4. Do not use SD-61 at reverse rotation.
5. Do not use at unusual angle and excessive pressure.
6. Insert the shank into the handpiece according to the manufacturer's instructions.
7. Wear safely glasses to protect your eyes.