

# SAF SYSTEM RDT3 Compatibility Guide

## RDT3-NX Handpiece Head

RDT3-NX

NSK - type Handpiece

E-Type micro-motor  
set to 5,000 RPM




## RDT3 Handpiece Head

RDT3

KaVo - type Handpiece

E-Type micro-motor  
set to 5,000 RPM

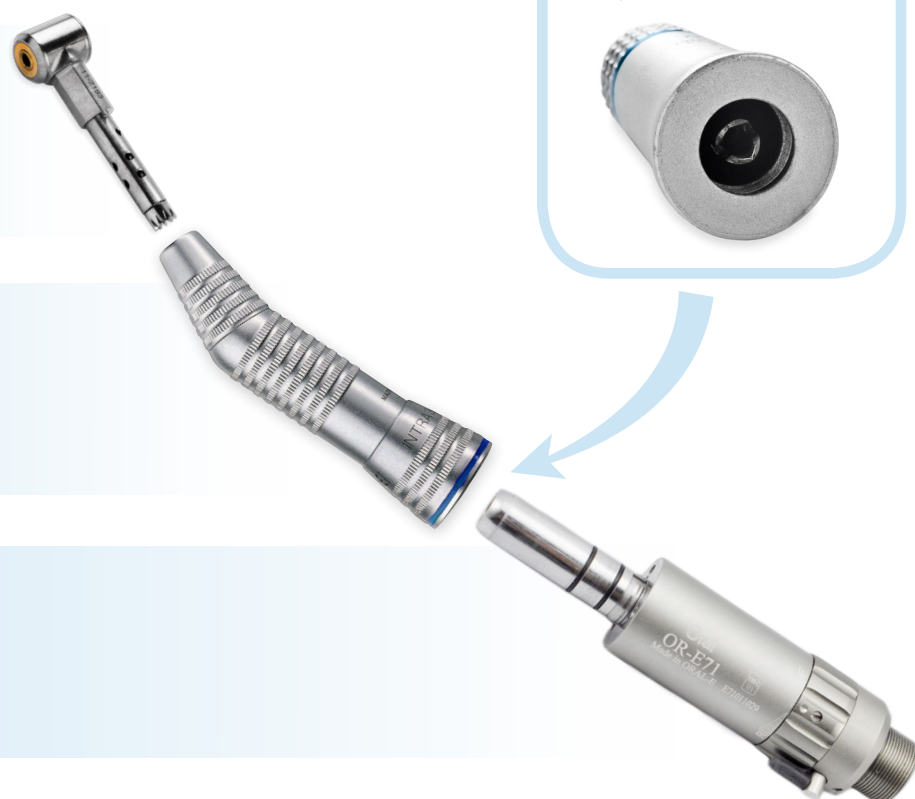

**EndoStation**  
ReDent Nova by Acteon



**WaveOne**  
Dentsply



**Reciproc (Gold/Silver)**  
VDW



**Dental Unit**  
With digital speed control



**Endopilot**  
Schlumbohm



**EndoCenter**  
Acteon



**Endodontic  
Micromotor**  
(various)



**Physiodispenser**  
(various)

# SAF SYSTEM RDT3 Compatibility Guide

## RDT3-NX Handpiece Head + Adapter

**RDT3-NX**



**F-Type adapter required:**

**NSK FC C873 Direct drive (1:1) quick-coupling**



**Additional motors may be compatible.**

**Speed settings: 5000 rpm (calibrated) - note gear ratio settings**

**Torque control: off / max.**

Various motors may require specific settings - consult with an authorized technician.



**EndoMate DT**  
NSK



**X-Smart**  
Dentsply



**EndoSequence**  
Brasseler



**EndoMate TC**  
NSK



**X-Smart Easy**  
Dentsply



**EndoTouch TC/TC2**  
SybronEndo



**EndoSequence II**  
Brasseler



**EndoMate TC2**  
NSK



**CanalPro CL**  
ColteneEndo