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PRODUCT IDENTIFICATION

This manual is an integral part of the product. For any correspondence concerning the product, please contact:

SILFRADENT

Via G. Di Vittorio, 35-37
47018 S. Sofia (Forlì) Italia
tel. #39 543 970684
fax #39 543 970770

hereby known as the "manufacturer", quoting the details given on the registration label located on the back of the equipment, and stating the date of purchase.

Caution

If the registration plate should be removed, the guarantee will automatically become null and void. However, if the registration plate should become illegible, please ask for another copy from **Silfradent**.

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PREFACE

Introduction

The state-of-the-art equipment you have purchased has been designed and built in compliance with the Directive 89/32/CEE and subsequent modifications, for the part of which such regulations apply.

SILFRADENT wishes to thank you and congratulate you for having chosen its products and is happy to be able to include you amongst its numerous customers.

The product you have purchased has been designed by highly qualified staff with many years experience in the sector.

The construction underlines the high standards of quality that have always distinguished our products.

This document will enable you not only to become familiar with the equipment you have bought, but also to full advantage.

Therefore, we kindly ask you to read the following pages carefully and to follow the indications and advice given.

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PURPOSE AND CARE OF THE MANUAL

SILFRADENT has drawn up this manual in the aim of assisting the user of the equipment and in order to facilitate the setting up and use of the same.

It has not been written for the purpose of training the operator to do the work to be carried out.

Each piece of equipment is supplied with the copy of this manual and the instruction contained in it must be complied with in order to ensure a safe and perfect operation of the device.

Any instruction that **SILFRADENT** has purposely omitted, refer to operations that must only be carried out by a specialized technician authorized by the manufacturer.

Before carrying out any operation on the machine, the operator must first read this manual carefully and understand fully the contents of each chapter.

Even though this manual has been thoroughly checked in order to guaranteed the utmost precision, we kindly ask you to contact us if you should have any doubts or in the case of imprecise descriptions or omissions.

For any controversy that may arise as a result of an unclear translation, the manual written in Italian will be binding.

Text appearing in boxes like this require maximum attention as they contain warnings of danger that must be read with great care

This manual cannot replace the knowledge of an expert operator, but it provides a reminder of the main operations, although, if necessary, a person with specific qualifications in this field should be consulted.

The manual forms as integral part of the equipment and must be kept for future reference until the equipment is finally dismantled.

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Keep the manual in a safe, dry place, close the equipment so than it can be consulted whenever necessary. If the manual should be damaged, or lost, a further copy should be requested directly from the manufacturer.

The previous manual supplied was based on the equipment as it was when it was marketed. The new manual may provide greater information owing to the technological progress of the product, but the old manual **CANNOT** be considered unsuitable for this reason.

The manufacturer is no obliged to update previous products or manuals and even to acknowledge economic damages due to machine standstills.

If the user should sell the equipment, then he must provide the manufacturer with the name and address of the purchaser in order to enable any supplements to the manual to be sent to the new owner

Caution

The figures show in this manual are only given as guidelines and are of a general nature. The specific operations to be carried out on the individual models will be dealt with case by case.

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WARNINGS

IMPORTANT !!!

Silfradent has drawn up this manual in the aim of assisting and in order to facilitate the setting up and use of the instrument.

The manufacturer cannot accept any responsibility for damage resulting from improper use or modification made to the equipment.

Each EVOLUTION EV1 steam machine is supplied with a copy of this manual and the instructions contained in it must be complied with in order to ensure a safe and perfect operation of the device. The manual is an integral part of the equipment and must be kept for future reference until the equipment is finally dismantled.

If the user should not consider these instructions, the warranty conditions will cease automatically.

For any type of request you are kindly advised to specify the model and the serial number put on the back label.

The lack of label makes cease the warranty condition.

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INTRODUCTION

The current manual is dedicated to the steam machine at 4 bar with the code EV1 SJ, which is part of our EVOLUTION line.

Silfradent wishes to thank you and congratulate you for having chosen its products and is happy to include you amongst its numerous customers.

This document will enable you to not only become familiar with the equipment you have bought, but also to use it to full advantage.

When the equipment is delivered, make sure that the packaging has not been damaged during transportation.

Even if all necessary precautions are taken during packing phase and during the preparation for transportation to prevent damage during delivery, it is important to check the integrity of the equipment when it has been removed from the packaging. If any damage should be noted, a complaint should be sent to the supplier immediately.

It is advisable to keep all the equipment packaging material intact in case the machine has to be returned for an inspection or servicing.

CAUTION ! Make sure the guarantee certificate is included in the pack and that it has been stamped and filled in by the warehouse and that copies have been retained by the warehouse and by Silfradent.

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CONTENTS OF PACK

- Steam machine
- Operating and maintenance manual
- Guarantee certificate
- Container for water filling

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PRECAUTIONS

IMPORTANT !!! The steam machine EV1 SJ is an instrument to be used only in dental laboratories. Any other use is to be considered improper. The manufacturer can not be held responsible if the product is used under any other condition.

The equipment must be used by competent and qualified operators who understand techniques related to this instrument.

Bearing in mind that the equipment is supplied a mains voltage 220 Volts, all the appropriate safety measures must be followed.

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DESCRIPTION

The Steam machine Evolution EV1 SJ is the result of a strenght collaboration with the most modern dental laboratories and of a precise study of the new technologies in mechanic and electronic branches.

This machine enable to resolve in an easy and fast way, all the problems related to decontamination of metal castings, the elimination of investment residue, washing burrs and all types of cleaning procedures.

The main technical characteristics are the following :

- Boiler constructed in antideflagrant material
- Manometer with pressure indication
- Switch informing about the lack of water
- Switch for pre-heating
- Pressure at 4.5 bar
- Boiler capacity 4 litres
- Maximum usable capacity 3 litres
- Electro-magnet general switch
- Mobile spray gun with low voltage command
- Knob for flow control
- Security Cover (7 bar)
- Thermostat for resistance check

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INSTALLATION

Put the machine on a stable table in an easily reachable position. Then unscrew the superior cover and introduce about 3 litres of distilled water, till the moment you will notice a water overflow.

Finally connect the feed cable to an earthed mains socket at 220 Volts.

**CAUTION !! USE ONLY DISTILLED
WATER !!!!!**

**CAUTION !! Check that the wiring system is
suitably earthed.**

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OPERATIVE PROCEDURES

Once the end user has connected the feed cable to the mains socket, it is possible to start up the machine through the general switch placed on the front part.

The pre-heating green light will immediately be illuminated. You will notice that the manometer will start rising till 4,5 bar (around mid of the pressure gauge azure zone). At that moment the green pre-heating light will switch off.

From this phase it is impossible to take off the cover and consequently add water, because inside the boiler the steam pressure starts on working.

CAUTION ! ! ! The steam could be hot and dangerous for the end user or those in the vicinity.

When the "lack of water" red light is illuminated and sound alarm is activated, the machine automatically switches off. Wait until the manometer will show pressure 0.

Then try to open the superior security cover. Should you still notice steam flowing out, wait for a while.

The steam can be obtained by means of white switch placed on pistol, and can be regulated with the front panel knob.

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PRESERVATION MACHINE**Quartely Cleaning & Maintenance Program**

1. Use 50 cc of Cleaning solution every 3 months (or more often, if necessary);
2. Turn power off, unplug and allow steam cleaner to fully cool down;
3. Slowly release all pressure remaining in steam cleaner;
4. Carefully remove steam cleaner screw cap;
5. Add distilled water until steam cleaner is approx. 75% full;
6. Puor in 50 cc of Cleaning solution;
7. Secure steam cleaner's screw cap back into place;
8. Plug-in , turn on and allow steam cleaner to heat-up for 15min;
9. Point spray handle down & press the steam button;
10. Turn power off, unplug and allow steam cleaner to cool overnight;

Over a sink carefully unscrew drain nut located on bottom of steam cleaner. Remove screw cap, rinse by running water in through top and draining out from bottom for 2-3 minutes before securing drain nut back into place. Fill with distilled water. Replace screw cap O-ring.

CAUTION : For external use only. Do not swallow. Avoid contact with skin and eyes. Wear eye protection and rubber gloves. Do not mix or combine with any other cleaning agent(s) or chemical (s). Dispose of bottle after use. For professional use only. Use only as directed. Read all instructions, and MSDS prior to use.

KEEP OUT OF REACH OF CHILDREN.

If the equipment should be not used for long periods, it should be store d in a protected ad dry place.

It should be covered with a suitable cloth in order to protect it from dust and humidity.

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DEMOLITION MACHINE

The materials used to built the equipment are , in part, environmentally-friendly and therefore, several precaution must be taken when the product is sent for scrap.

Divide the materials according to their types (e.g. ferrous materials, rubber, plastic etc.).

As some of the components of the equipment are very small in size, they may constitute secondary hazards in the time prior to being sent for scrap.

Caution

Do not leave the components unattended or within reach of children or animals as they could be hazardous.

When disposing of the materials or sending them for scrap, follow the laws in force in the Country, contacting recognized and authorized companies specializing in such work, if necessary.