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**INDEX**

PRODUCT IDENTIFICATION .....	3
PREFACE .....	4
PURPOSE AND CARE OF THE MANUAL .	5
PRESERVATION MACHINE .....	7
CAUTION .....	7
INTRODUCTION .....	8
EQUIPMENT .....	8
PRECAUTIONS .....	9
DESCRIPTION .....	10
INSTALLATION .....	10
USE AND MAINTENANCE .....	11
MAINTENANCE PROGRAMME .....	11
MAIN TECHNICAL PROBLEMS .....	11
SECURITY INSTRUCTIONS .....	12
CAUTION !!! .....	12
TECHNICALS FEATURES .....	12
DEMOLITION MACHINE .....	13

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This manual is an integral part of the product. For any correspondence concerning the product, please contact:

#### **SILFRADENT**

Via G. Di Vittorio, 35-37  
47018 S. Sofia (Forlì)  
Italia tel. #39 543 970684  
fax #39 543  
970770

hereby known as the "manufacturer", quoting the details given on the registration label located on the back of the equipment, and stating the date of purchase.

#### **Caution**

If the registration plate should be removed, the guarantee will automatically become null and void. However, if the registration plate should become illegible, please ask for another copy from **Silfradent**.

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##### Introduction

The state-of-the-art equipment you have purchased has been designed and built in compliance with the Directive 89/32/CEE and subsequent modifications, for the part of which such regulations apply.

**SILFRADENT** wishes to thank you and congratulate you for having chosen its products and is happy to be able to include you amongst its numerous customers.

The product you have purchased has been designed by highly qualified staff with many years experience in the sector.

The construction underlines the high standards of quality that have always distinguished our products.

This document will enable you not only to become familiar with the equipment you have bought, but also to full advantage. Therefore, we kindly ask you to read the following pages carefully and to follow the indications and advice given.

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### PURPOSE AND CARE OF THE MANUAL

**SILFRADENT** has drawn up this manual in the aim of assisting the user of the equipment and in order to facilitate the setting up and use of the same.

It has not been written for the purpose of training the operator to do the work to be carried out.

Each piece of equipment is supplied with the copy of this manual and the instruction contained in it must be complied with in order to ensure a safe and perfect operation of the device.

Any instruction that **SILFRADENT** has purposely omitted, refer to operations that must only be carried out by a specialized technician authorized by the manufacturer.

Before carrying out any operation on the machine, the operator must first read this manual carefully and understand fully the contents of each chapter.

Even though this manual has been thoroughly checked in order to guaranteed the utmost precision, we kindly ask you to contact us if you should have any doubts or in the case of imprecise descriptions or omissions.

For any controversy that may arise as a result of an unclear translation, the manual written in Italian will be binding.

**Text appearing in boxes like this require maximum attention as they contain warnings of danger that must be read with great care**

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This manual cannot replace the knowledge of an expert operator, but it provides a reminder of the main operations, although, if necessary, a person with specific qualifications in this field should be consulted.

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The manual forms as integral part of the equipment and must be kept for future reference until the equipment is finally dismantled.

Keep the manual in a safe, dry place, close the equipment so than it can be consulted whenever necessary. If the manual should be damaged, or lost, a further copy should be requested directly from the manufacturer.

The previous manual supplied was based on the equipment as it was when it was marketed. The new manual may provide greater information owing to the technological progress of the product, but the old manual **CANNOT** be considered unsuitable for this reason.

The manufacturer is no obliged to update previous products or manuals and even to acknowledge economic damages due to machine standstills.

If the user should sell the equipment, then he must provide the manufacturer with the name and address of the purchaser in order to enable any supplements to the manual to be sent to the new owner

#### **Caution**

The figures show in this manual are only given as guidelines and are of a general nature. The specific operations to be carried out on the individual models will be dealt with case by case.

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## PRESERVATION MACHINE

The product does not require any particular care or attention in order to be kept in good operating order.

The coated parts guarantee resistance to corrosive agents and the maintenance of the physical and structural characteristics.

If the equipment should be not used for long periods, it should be stored in a protected and dry place.

It should be covered with a suitable cloth in order to protect it from dust and humidity.

## CAUTION

### **IMPORTANT!!!**

SILFRADENT has compiled this booklet to help you handle and operate the equipment better.

SILFRADENT cannot accept any responsibility for damage resulting from incorrect use or modifications made to the equipment.

A copy of this booklet accompanies each machine. To ensure correct instrument operation, please make sure you follow carefully the instructions provided.

The machine must never be used for other purposes than those indicated in the operating instructions and leaflet.

Non-compliance with the instructions contained in this booklet will result in the immediate termination of the guarantee.

On all requests, always indicate the date of purchase, the machine model and the serial number shown on the plate located on the back of the machine. Removal of this plate will result in the immediate termination of the guarantee.

All the technical details have to be intended as pure indicative information. The company Silfradent declares the right to modify the present booklet without any compulsory immediate advice to the customer.

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## INTRODUCTION

This manual refers to the hydraulic press 660. **SILFRADENT congratulates and thanks you for your choice !!**

The following instructions will help you get to know and use the machine better.

Always make sure the certificate of guarantee is attached to the machine and that this has been completed and stamped by the stockists. A copy of this guarantee should remain with the stockists and with SILFRADENT.

Before unpacking the machine, use the following check list to ensure no component parts are missing.

After carefully removing all parts from their packing, place these on a table to ensure they have not been damaged during transport. If any parts are found to be damaged, contact your stockists immediately. Also make sure that all packing residue is removed from the machine.

Now follow the installation instructions in this booklet.

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## PRECAUTIONS

**IMPORTANT** -The Hydraulic Press is a dental laboratory instrument.

**IMPORTANT** -No other possible uses exist for this instrument outside the dental sphere.

The instrument must be used by qualified personnel who are familiar with the uses for which the instrument is intended.

Please make sure you followed the following instructions :

- Install the instrument on a plane and stable surface.
- Don't use the machine as soon as you realize a damage.

This instrument creates kg. 15.000 maximum effort, as consequence it must be handled with high care otherwise it would also be hazardous for the user.

Don't never try to block the press thrust with an foreign body or with the hands, it would be very dangerous and there is also the risk to damage the equipment.

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## DESCRIPTION

The Hydraulic Press consists of a base containing hydraulic oil (SAE10). This oil has the function to create an up movement to the piston which, in its turn, create a pressure on the flasks positioned under the hydraulic press. The movement of the piston is indeed enabled by the lever on the mechanic pump. The maximum piston's stroke is 2.5 cm, as consequence it must be adapted to the number of flasks. This distance can be regulated by the mean of the chromated screw; the maximum reachable distance is around 25 cm.

Since the operator has fixed the pressure, the instrument maintains always the chosen value. **CAUTION!** Pay attention to the below table of recommended efforts otherwise you risk to damage the flasks.

In order to eliminate the pressure, unscrew the tap on lateral basement's side, which enables a rapid piston release.

## INSTALLATION

Though during packing and preparation for shipment, all precautions have been taken to prevent damage during transport, it is important to check the condition of the machine after removing it from the packing. In the event of any damage, immediately notify the supplier.

It is best not to throw away the packaging as this may be useful if the machine has to be sent back for checking or overhauling.

**IMPORTANT** - Don't never put the instrument upside-down, otherwise you can create an oil leak.

The machine is delivered ready to be used, it is sufficient to screw on the lever into its position on the mechanic pump.

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## USE AND MAINTENANCE

The simple mechanic structure of the instrument doesn't require any special maintenance procedure. The only caution is to follow the tables with effort value recommended by the manufacturer of flasks. Given the notable force of the 660 hydraulic press, it is sufficient to operate between 4000 and 7000 Kg. of thrust. These values provide an uniform flow and an excellent works result, without damaging the flasks.

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## MAINTENANCE PROGRAMME

As above mentioned the simple structure doesn't require any special maintenance program. The machine can work with each type of hydraulic oil (SAE 10). Please find herewith attached the copy of the exploded instrument view.

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## MAIN TECHNICAL PROBLEMS

A) What to do if the hydraulic press doesn't maintain anymore the same pressure ? ? ? ?

- 1) Check both the O-ring on the mechanic pump and on the piston.
- 2) Check if the tap/faucet is leaking and if it is well fixed.

B) What to do if the pump doesn't create any pressure ? ? ? ?

- 1) Check if mechanic pump is creating movement to the inner piston.
- 2) Check if there is any block in the oil pipe net.

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### SECURITY INSTRUCTIONS

- 1) Before carrying out any operation on the hydraulic's net, we always recommend to contact us or a specialized center.
- 2) Don't modify the pressure gauge
- 3) Always verify the level of oil be the mean of the special screw cap

### CAUTION !!!

If there is the need to repair the instrument, we recommend you to apply to specialized centers and to use original spare parts.

### TECHNICALS FEATURES

**Weight**  
Kg. 28  
**Width**  
22 cm  
**Depth**  
19 cm  
**Height**  
48 cm  
**Distance between the two columns**  
12 cm  
**Distance between the two plates**  
2.5 cm  
**Maximum capacity : till three flasks (Article 110) and one bracket (Article 263)**  
**Maximum pressure / effort**  
15000 Kg.

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### DEMOLITION MACHINE

The materials used to built the equipment are , in part, environmentally-friendly and therefore, several precaution must be taken when the product is sent for scrap.

Divide the materials according to their types (e.g. ferrous materials, rubber, plastic etc.).

As some of the components of the equipment are very small in size, they may constitute secondary hazards in the time prior to being sent for scrap.

#### **Caution**

Do not leave the components unattended or within reach of children or animals as they could be hazardous.

When disposing of the materials or sending them for scrap, follow the laws in force in the Country, contacting recognized and authorized companies specializing in such work, if necessary